

# Installation Manual

CuratOR™  
EX3140

Color LCD Monitor

## Important

**Please read this “Installation Manual”, and “Instructions for Use” (separate volume) carefully to familiarize yourself with safe and effective usage.**

Refer to the “Instructions for Use” for information on the installation / connection of the monitor.



This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

---

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

---

# CONTENTS

<b>CONTENTS</b>	<b>3</b>
<b>Chapter 1 Basic Adjustment / Setting</b>	<b>4</b>
1-1. Loading the Monitor Settings	4
1-2. Switching Input Signals	5
1-3. Using Function Buttons	5
<b>Chapter 2 Advanced Adjustment / Setting</b>	<b>6</b>
2-1. Operation via the Main Menu	6
2-2. Individual Functions of Main Menu	7
● Video Config	7
● System Config	8
● Window Config	10
● PRESET	11
● Control	11
<b>Chapter 3 Troubleshooting</b>	<b>13</b>
3-1. No Picture Displayed	13
3-2. Imaging Problems	14
3-3. Other Problems	14
<b>Chapter 4 Reference</b>	<b>15</b>
4-1. Main Default Settings	15
4-2. Pin Specifications	16
<b>Appendix</b>	<b>17</b>
LIMITED WARRANTY	17
Trademark	18
License	18
Recycling Information	19

# Chapter 1 Basic Adjustment / Setting

This chapter describes the basic functions that can be adjusted and set by pressing the buttons on the front of the monitor.

## 1-1. Loading the Monitor Settings

It is possible to switch to PRESET, FACTORY, or the USER settings that have been set in advance.

PRESET	Basic settings are made in advance at the factory. The settings of PRESET cannot be changed.
FACTORY	Returns to settings when shipped from the factory.
USER	Various settings including menus can be made for personal use. For details, see <a href="#">“PRESET” (page 11)</a> .

### 1. Press (PRESET).

The preset menu is displayed.




### 2. Press to select the PRESET number, FACTORY, or USER number to load, and then press (ENTER).

The pre-made settings are loaded and the settings are changed. The selectable settings are as follows.

- PRESET1 to PRESET5
- FACTORY
- USER1 to USER10

#### Note


- The “PRESET” menu will disappear after five seconds have elapsed if no operation is performed.
-  is displayed for the USER number that has already been registered.
- When selecting a USER number that has not yet been registered, “No Data” is displayed.
- For information about the main default settings, see [“4-1. Main Default Settings” \(page 15\)](#).

## 1-2. Switching Input Signals

Select the signal input to the monitor and display it on the screen.

### 1. Press (INPUT).

The “Input Select” menu appears.

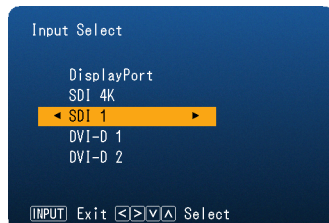




### 2. Press to select an input signal.

This switches to the selected input signal.



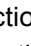
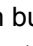

### 3. Press (INPUT).

The menu screen exits.

#### Note


- When selecting SDI, SDI 1 to SDI 4 can be selected using the   buttons.
- If you press  (INPUT) while the “Input Select” menu is displayed in dual screen mode, the “Input Select” menu of the subscreen is displayed.
- The Input Select menu will display for 7 seconds without input before it disappears.

## 1-3. Using Function Buttons

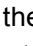
Execute the functions assigned to the function buttons (F1 (  ), F2 (  ), F3 (  ), F4 (  ) and F5 (  )). For information on how to set a function to each button, see “Function” (page 8).

### 1. Press F1 ( ), F2 ( ), F3 ( ), F4 ( ) or F5 ( ).

When the function menu is displayed, the function associated with the button that is pressed will run.

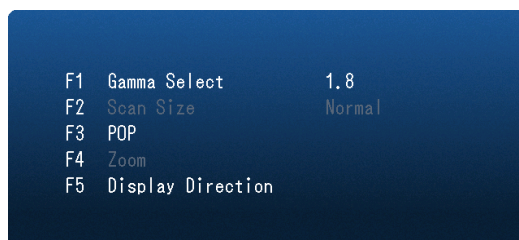

#### Note

- In dual screen mode, press  (INPUT) to select a screen for performing the target function.
- When the function button for a function that cannot be run is pressed, the message “Invalid Function” will appear.
- When the function button that has not yet been defined is pressed, the message “Undefined” will appear.

# Chapter 2 Advanced Adjustment / Setting

This chapter describes advanced monitor adjustment and setting procedures using the main menu.

## 2-1. Operation via the Main Menu




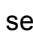


### 1. Menu display

1. Press  (MENU).  
The main menu is displayed.



2. Press   to select the desired item, and then press  .

### 2. Adjusting / setting

1. Press   to select the desired item.
2. Press   to set.  
If "Next Menu" is displayed for items, press  , and a next menu is displayed.



### 3. Exiting

1. If you repeatedly press  (MENU), the menu screen exits.

## 2-2. Individual Functions of Main Menu

### ● Video Config

Adjustments / settings for image display.



Function	Setting range	Description
Color	0 to 100	Adjusts color depth of the image.
Tint	0 to 100	Adjusts skin tones.
Brightness	0 to 100	Adjusts image darkness (black).
Picture	0 to 100	Adjusts image intensity.
S.R.S.C	0 to 20	This adjusts picture crispness.
Gamma	1.8 to 2.6 DICOM HLG PQ	Adjusts brightness.
Backlight	0 to 100	Adjusts brightness of backlight. In case of multiple windows, select the main screen. The setting cannot be changed when a screen other than the main screen is selected.
Color Temp	6500 K 9300 K Off	Switches tone of the image.
R Gain	0 to 100	Adjusts the brightness of red in bright portions.
G Gain	0 to 100	Adjusts the brightness of green in bright portions.
B Gain	0 to 100	Adjusts the brightness of blue in bright portions.
R Bias	0 to 100	Adjusts the brightness of red in dark portions.
G Bias	0 to 100	Adjusts the brightness of green in dark portions.
B Bias	0 to 100	Adjusts the brightness of blue in dark portions.
Color Gamut	sRGB Native BT2020	Sets the Color Gamut.
Input Color Format	RGB YUV422 YUV444	When there is a DisplayPort input, this sets the color format.
Input Range	0 to 255 16 to 235 16 to 255	Changes the dynamic range depending on the input signal. <ul style="list-style-type: none"> <li>• “0 to 255” Use this setting when connecting PC format source.</li> <li>• “16 to 235” Set when connected to a video format device or PC format device.</li> <li>• “16 to 255” Set when connected to a video format device or PC format device.</li> </ul>
Noise Reduction	Off Light Middle Heavy	Select Noise Reduction level for picture.

## ● System Config

Settings for devices.




Function		Setting range	Description
SDI 4K		Auto SQUARE INTERLEAVE	Sets the signal format. <ul style="list-style-type: none"> <li>• “Auto” SQUARE or INTERLEAVE are automatically detected.</li> <li>• “SQUARE” Treats as Square Division (SQD) signal.</li> <li>• “INTERLEAVE” Treats as Two-Sample Interleave Division (2SI) signal.</li> </ul>
Function	F1 F2 F3 F4 F5	POP Display Direction Scan Size Gamma Select Color Gamut Noise Reduction Zoom Undef	Sets functions assigned to function buttons. <ul style="list-style-type: none"> <li>• “POP” Switches the screen display between the dual screen (P out P: a screen is split vertically) and the single screen.</li> <li>• “Display Direction” Rotates / flips the screen.</li> <li>• “Scan Size” Switches the setting values of “Window Config” - “Scan Size” in sequence.</li> <li>• “Gamma Select” Switches the setting values of “Video Config” - “Gamma” in sequence.</li> <li>• “Color Gamut” Changes the color gamut.</li> <li>• “Noise Reduction” Changes between the values “Video Config” and “Noise Reduction”.</li> <li>• “Zoom” Expands the current view on the display.</li> <li>• “Undef” No function assigned.</li> </ul>
Status Display		Continue 3Sec Off	Sets whether information about the current input signal and terminal name is displayed. Information about the resolution at the PC signal input is also displayed.
Power On Setup		Last PRESET1 to PRESET5 FACTORY USER1 to USER10	Settings for switching on can be set in advance. <ul style="list-style-type: none"> <li>• “Last” Starts with last saved state.</li> <li>• “PRESET1 to PRESET5” State saved in PRESET is called. Five types of PRESET are stored in advance in the monitor as fixed data.</li> <li>• “FACTORY” Returns to factory settings.</li> <li>• “USER1 to USER10” State saved in USER is called. It is possible to store 10 types of data in USER.</li> </ul>



Function	Setting range	Description
Power Save Mode	On Off	Sets whether function that the backlight turns off due to no video signal after 30 sec is valid.
Language	English Deutsch Français 日本語 Italiano Español Svenska 簡体中文 繁體中文	Sets the language to be used for display in settings and all menus.
Logo Display	On Off	The logo display on power-on can be set to On or Off.

## ● Window Config


Settings for screen display.

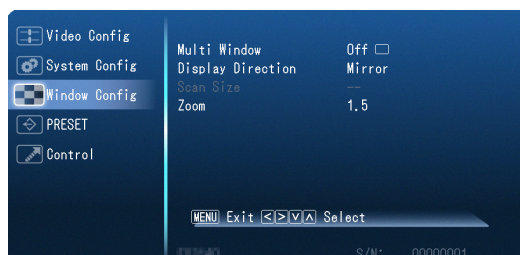

Function	Setting range	Description
Multi Window	Off <input type="checkbox"/> 2POP <input type="checkbox"/>	Sets dual screen display. <ul style="list-style-type: none"> <li>• “Off <input type="checkbox"/>” Sets single screen display mode</li> <li>• “2POP <input type="checkbox"/>” Displays split window (P out P).</li> </ul>
Display Direction	Normal 180° Mirror	Sets the rotation angle or horizontally flips the main screen and subscreen. <p><b>Note</b></p> <ul style="list-style-type: none"> <li>• When “180°” is selected <input type="checkbox"/>, it will be displayed on the screen.</li> <li>• When “Mirror” is selected <input type="checkbox"/>, it will be displayed on the screen.</li> </ul>
Scan Size	(When Multi Window mode is set to “Off”) Normal Through  (When Multi Window mode is set to “2POP”) H Fit V Fit	Sets zoom in and out ratios when displaying input video on the monitor. <ul style="list-style-type: none"> <li>• “Normal” When using the single screen, enlarges or reduces so that the vertical resolution is 2160 dots. Dark bands form left and right when the aspect ratio is other than 16:9.</li> <li>• “Through” Dot-by-dot display.</li> <li>• “H Fit” The input signal will fill the POP horizontally.</li> <li>• “V Fit” Displays the input signal matching the vertical display frame of the POP.</li> </ul>
Zoom	1.5 2.0	Setting for the zoom percentage when an image is zoomed. <ul style="list-style-type: none"> <li>• “1.5” Expand by 1.5.</li> <li>• “2.0” Expand by 2.0.</li> </ul>

## ● PRESET

Settings for “USER1” to “USER10”.

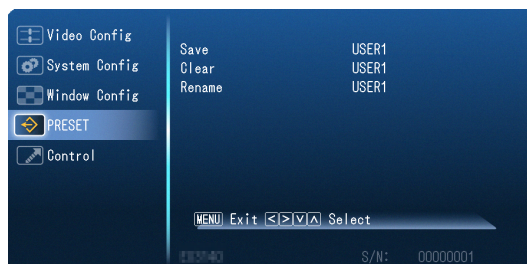















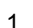








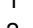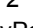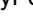









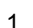


Function	Description
Save	Saves currently set values, and the status for each user. <b>Attention</b> • “Power On Setup”, “Language” and “Logo Display” settings cannot be registered.
Clear	The set values of “USER1” to “USER10” (USER NAME) can be deleted.
Rename	The character strings of “USER1” to “USER10” (USER NAME) can be changed.

## ● Control

Settings for the lock function and control functions when this device is controlled by an external device.

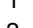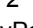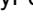



Function	Setting range	Description
Lock	On Off	Changing of settings via the main menu is prevented.
COM Port	USB RS-232C Off	Settings for the port when used as a COM port.
GPI Control	On Off	Sets the On / Off status of the remote function via the GPI terminal.

Function		Setting range	Description
GPI Setup	GPI1	Zoom	Sets functions assigned to each terminal of the GPI terminals. <ul style="list-style-type: none"> <li>• “Zoom” Switch between the set zoom level and actual size.</li> <li>• “POP” Switches between dual screen (P out P) and single screen.</li> <li>• “Display Direction  Rotates or flips main screen.</li> <li>• “Display Direction  Rotates / flips subscreen.</li> <li>• “ Input DisplayPort” Switches main screen input signal to DisplayPort.</li> <li>• “ Input SDI 4K” Switches main screen input signal to SDI 4K.</li> <li>• “ Input SDI 1” Switches main screen input signal to Input SDI 1.</li> <li>• “ Input SDI 2” Switches main screen input signal to Input SDI 2.</li> <li>• “ Input SDI 3” Switches main screen input signal to Input SDI 3.</li> <li>• “ Input SDI 4” Switches main screen input signal to Input SDI 4.</li> <li>• “ Input DVI-D 1” Switches main screen input signal to Input DVI-D 1.</li> <li>• “ Input DVI-D 2” Switches main screen input signal to Input DVI-D 2.</li> <li>• “ Input DisplayPort” Switches subscreen input signal to DisplayPort.</li> <li>• “ Input SDI 1” Switches subscreen input signal to Input SDI 1.</li> <li>• “ Input SDI 2” Switches subscreen input signal to Input SDI 2.</li> <li>• “ Input SDI 3” Switches subscreen input signal to Input SDI 3.</li> <li>• “ Input SDI 4” Switches subscreen input signal to Input SDI 4.</li> <li>• “ Input DVI-D 1” Switches subscreen input signal to Input DVI-D 1.</li> <li>• “ Input DVI-D 2” Switches subscreen input signal to Input DVI-D 2.</li> <li>• “Red Tally” Sets  (red) displayed at screen top center to On or Off.</li> <li>• “Green Tally” Sets  (green) displayed at screen top center to On or Off.</li> <li>• “Undef” Not set.</li> </ul>
	GPI2	POP	
	GPI3	Display Direction 	
	GPI4	Display Direction 	
	GPI5	 Input DisplayPort	
	GPI6	 Input SDI 4K	
	GPI7	 Input SDI 1	
	GPI8	 Input SDI 2	
		 Input SDI 3	
		 Input SDI 4	
		 Input DVI-D 1	
		 Input DVI-D 2	
		 Input DisplayPort	
		 Input SDI 1	
		 Input SDI 2	
		 Input SDI 3	
		 Input SDI 4	
		 Input DVI-D 1	
		 Input DVI-D 2	
		Red Tally	
		Green Tally	
		Undef	

#### Attention



- When the LOCK function is “On”, only “Control” settings can be changed.

#### Note

- The GPI is an edge operation without tally. Operations can be received by setting the terminal potential from open to the same potential as GND. The edge operation requires a period of at least 200 ms of the same potential as GND. After the operation, the same potential as GND should be canceled.
- If you hold down  and  for approximately four seconds, buttons are locked, and the “LOCKED” message is displayed. If you perform the same operation again, buttons are unlocked, and the “UNLOCKED” message is displayed.

# Chapter 3 Troubleshooting



## 3-1. No Picture Displayed

Problem	Possible cause and remedy
<b>1. No picture.</b>	<ul style="list-style-type: none"><li>• Check whether the power cord is connected properly.</li><li>• Check whether the DC OUT terminal and the DC IN terminal are connected properly.</li><li>• Turn on the power switch.</li><li>• Check whether the main power indicator of the AC adapter is on.</li><li>• Turn off the power and then turn it on again.</li></ul>
<b>2. The message below appears.</b>	This message appears when the signal is not input correctly even though the monitor is functioning properly.
<ul style="list-style-type: none"><li>• This message appears when no signal is input. Example: </li></ul>	<ul style="list-style-type: none"><li>• The message shown on the left may appear, because some devices to be connected do not output the signal immediately after power-on.</li><li>• Check whether the device to be connected is turned on.</li><li>• Check whether the signal cable is connected properly.</li><li>• Turn off the power and then turn it on again.</li></ul>
<ul style="list-style-type: none"><li>• The message indicates that the input signal is outside the specified frequency range. Example: </li></ul>	<ul style="list-style-type: none"><li>• Check whether the device to be connected is configured to meet the resolution and vertical scan frequency requirements of the monitor.</li><li>• Reboot the device to be connected.</li></ul>

## 3-2. Imaging Problems

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none"> <li>Use “Brightness” or “Backlight” in the Setting menu to adjust it.</li> </ul>
2. Text is blurred	<ul style="list-style-type: none"> <li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see “Compatible Resolutions” in the Instructions for Use).</li> </ul>
3. Afterimages appear	<ul style="list-style-type: none"> <li>Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time.</li> <li>Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.</li> <li>An afterimage may appear even after a short time period has elapsed depending on the displayed image. If this occurs, changing the image or leaving the power off for a few hours may solve the problem.</li> </ul>
4. Green, red, blue, white dots or dark dots remain on the screen	<ul style="list-style-type: none"> <li>This is due to LCD panel characteristics and not a malfunction.</li> </ul>
5. The screen suddenly went black. / The screen does not become brighter after increasing the “Backlight” value in the Settings menu.	<ul style="list-style-type: none"> <li>This product is equipped with a heat dissipation fan. If the fan is not operating properly, screen brightness can be limited in order to lower the internal temperature. In this case, please make sure that the fan is operating properly.</li> </ul>

## 3-3. Other Problems

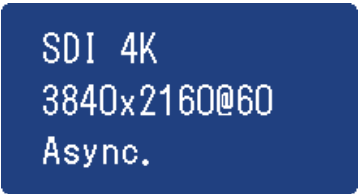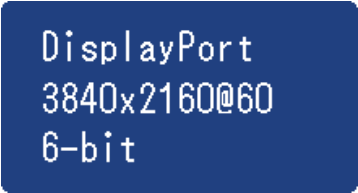Problem	Possible cause and remedy
1. The Setting menu does not appear.	<ul style="list-style-type: none"> <li>Make sure the lock function is not set to “On.” (See “Control” (page 11).)</li> </ul>
2. The message below appears.	This will be displayed when the signal is not displaying properly.
<ul style="list-style-type: none"> <li>When receiving SDI 4K input, this will display when a non-standard UHD signal is being input. Example:</li> </ul> 	<ul style="list-style-type: none"> <li>Turn off the power to the connected device and this product and then turn on the power again.</li> <li>Check the settings on the connected device.</li> </ul>
<ul style="list-style-type: none"> <li>This will be displayed when a 6-bit (64 gradient) signal is being input. Example:</li> </ul> 	<ul style="list-style-type: none"> <li>Turn off the power to the connected device and this product and then turn on the power again.</li> <li>Check the settings on the connected device.</li> </ul>

## Chapter 4 Reference

### 4-1. Main Default Settings

#### PRESET / FACTORY


	Input Select	
	Main Screen	Subscreen
<b>PRESET 1</b>	SDI 4K	DVI 1
<b>PRESET 2</b>	DVI 1	SDI 1
<b>PRESET 3</b>	DisplayPort	SDI 1
<b>PRESET 4</b>	SDI 4K	DVI 1
<b>PRESET 5</b>	DVI 1	SDI 1
<b>FACTORY</b>	DisplayPort	SDI 1

#### Others

System Configuration	Language	English
	Logo Display	On
Functions	F1	Gamma Select
	F2	Scan Size
	F3	POP
	F4	Zoom
	F5	Display Direction
Control Setting	Lock	Off


# 4-2. Pin Specifications

## ● GPI terminal

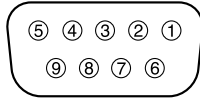


Pin No.	Signal name
1	GPI1
2	GPI2
3	GPI3
4	GPI4
5	GND
6	GPI5
7	GPI6
8	GPI7
9	GPI8

## ● DC IN terminal

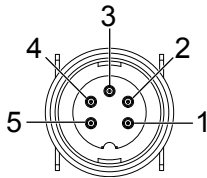

Pin No.	Signal name
1	GND
2	GND
3	F.E
4	48 VDC
5	48 VDC


## ● DC OUT terminal on the main unit

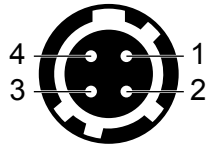

Pin No.	Signal name
1	GND
2	GND
3	5 VDC
4	5 VDC

HIROSE P/N: HR10A-7R-4S(74)  
 Recommended plug  
 HIROSE P/N: HR10A-7P-4P(74)

## ● DC OUT terminal of AC adapter

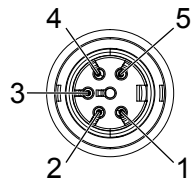

Pin No.	Signal name
1	GND
2	GND
3	F.E
4	48 VDC
5	48 VDC



# Appendix

## LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual of the Product (hereinafter referred to as the “**User’s Manual**”).

The period of this Warranty is two (2) years from the date of purchase of the Product (hereinafter referred to as the “**Warranty Period**”). EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts (excluding design parts) of the Product upon expiration of five (5) years after the production of the Product is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards. If the unit cannot be repaired due to its condition or the stockout of a relevant part, EIZO and Distributors may offer the replacement by a product with equivalent performance instead of repairing it.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor, including measurement value of the sensor;
- (d) Any defect of the Product caused by external equipments;
- (e) Any defect of the Product caused by the use under environmental conditions other than EIZO and Distributors’ estimation;
- (f) Any deterioration of the attachments of the Product (e.g. cables, User’s Manual, CD-ROM, etc.);
- (g) Any deterioration of the consumables, and/or accessories of the Product (e.g. batteries, remote controller, touch pen, etc.);
- (h) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel;
- (i) Any defect of the Product caused by placement in a location where it may be affected by strong vibration or shocks;
- (j) Any defect of the Product caused by liquid leaking from battery;
- (k) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in brightness, changes in brightness uniformity, changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.).

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

## Trademark

The terms HDMI and HDMI High-Definition Multimedia Interface, and the HDMI Logo are trademarks or registered trademarks of HDMI Licensing, LLC in the United States and other countries.

The DisplayPort Compliance Logo and VESA are registered trademarks of the Video Electronics Standards Association.

The SuperSpeed USB Trident Logo is a registered trademark of USB Implementers Forum, Inc.



DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

Kensington and Microsaver are registered trademarks of ACCO Brands Corporation.

Thunderbolt is a trademark of Intel Corporation in the United States and/or other countries.

Microsoft and Windows are registered trademarks of Microsoft Corporation in the United States and other countries.

Adobe is a registered trademark of Adobe Systems Incorporated in the United States and other countries.

Apple, macOS, Mac OS, OS X, Macintosh and ColorSync are registered trademarks of Apple Inc.

EIZO, the EIZO Logo, ColorEdge, DuraVision, FlexScan, FORIS, RadiCS, RadiForce, RadiNET, Raptor and ScreenManager are registered trademarks of EIZO Corporation in Japan and other countries.

ColorEdge Tablet Controller, ColorNavigator, CuratOR, EcoView NET, EIZO EasyPIX, EIZO Monitor Configurator, EIZO ScreenSlicer, G-Ignition, i•Sound, Quick Color Match, RadiLight, Re/Vue, Screen Administrator, Screen InStyle and UniColor Pro are trademarks of EIZO Corporation.

All other company and product names are trademarks or registered trademarks of their respective owners.

## License

The bitmap font used for this product is designed by Ricoh Industrial Solutions Inc.

# Recycling Information

## Recycling Information

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

<http://www.eizoglobal.com>

For recycling information for customers in Switzerland, please refer to the following website.

<http://www.swico.ch>

## Informationen zum Thema Recycling

Dieses Produkt muss gemäß den Vorschriften Ihres Landes zur Entlastung der Umwelt recyclet werden. Wenden Sie sich bei der Entsorgung dieses Produkts an einen Verteiler oder eine Tochtergesellschaft in Ihrem Land.

Die Adressen zur Kontaktaufnahme sind auf der unten angegebenen Website von EIZO aufgeführt.

<http://www.eizoglobal.com>

Kunden in der Schweiz entnehmen Informationen zum Recycling der folgenden Website:

<http://www.swico.ch>

## Informations sur le recyclage

Ce produit doit être jeté aux points de collecte prévus à cet effet et recyclé conformément à la législation de votre pays, afin de réduire l'impact sur l'environnement. Lorsque vous jetez ce produit, veuillez contacter un distributeur ou une société affiliée de votre pays.

Les adresses des distributeurs sont répertoriées sur le site Web EIZO ci-dessous.

<http://www.eizoglobal.com>

Pour les clients en Suisse, veuillez consulter le site Web suivant afin d'obtenir des informations sur le recyclage.

<http://www.swico.ch>

## Información sobre reciclaje

Este producto debe desecharse y reciclarse según la legislación del país para reducir el impacto medioambiental.

Cuando desee deshacerse de este producto, póngase en contacto con un distribuidor o una filial de su país.

Encontrará las direcciones de contacto en el sitio web de EIZO que se indica a continuación.

<http://www.eizoglobal.com>

## Informazioni sul riciclaggio

Per lo smaltimento e il riciclaggio del presente prodotto, attenersi alle normative vigenti nel proprio paese per ridurre l'impatto ambientale. Per lo smaltimento, rivolgersi ad un distributore o un affiliato presenti nel proprio paese.

Gli indirizzi sono elencati nel sito Web EIZO riportato di seguito.

<http://www.eizoglobal.com>

Per informazioni sul riciclaggio per i clienti in Svizzera, consultare il sito Web riportato di seguito.

<http://www.swico.ch>

## Återvinningsinformation

När denna produkt kasseras ska den hanteras och återvinnas enligt landets föreskrifter för att reducera miljöpåverkan.

När du kasserar produkten ska du kontakta en distributör eller representant i ditt land.

Kontaktadresserna listas på EIZO-webbplatsen nedan.

<http://www.eizoglobal.com>

## Πληροφορίες ανακύκλωσης

Το προϊόν αυτό, όταν απορρίπτεται, πρέπει να συλλέγεται και να ανακυκλώνεται σύμφωνα με τη νομοθεσία της χώρας σας έτσι ώστε να μην επιβαρύνει το περιβάλλον. Για να απορρίψετε το προϊόν, επικοινωνήστε με έναν αντιπρόσωπο ή μια θυγατρική εταιρεία στη χώρα σας.

Οι διευθύνσεις επικοινωνίας αναγράφονται στην τοποθεσία web της EIZO παρακάτω.

<http://www.eizoglobal.com>

## Сведения по утилизации

По истечении срока службы данного продукта его следует принести на сборный пункт и утилизировать в соответствии с действующими предписаниями в вашей стране, чтобы уменьшить вредное воздействие на окружающую среду. Прежде чем выбросить данный продукт, обратитесь к дистрибьютору или в местное представительство компании в вашей стране.

Контактные адреса можно найти на веб-узле EIZO.

<http://www.eizoglobal.com>

## Informatie over recycling

Wanneer u dit product wilt weggooien, moet het uit milieu-overwegingen worden verzameld en gerecycled volgens de betreffende wetgeving van uw land. Wanneer u dit product wilt weggooien, moet u contact opnemen met een distributeur of een partner in uw land.

De contactadressen worden vermeld op de volgende EIZO-website.

<http://www.eizoglobal.com>

## Informação sobre reciclagem

Este produto, quando o deitar fora, deve ser recolhido e reciclado de acordo com a legislação do seu país para reduzir a poluição. Quando deitar fora este produto, contacte um distribuidor ou uma filial no seu país.

Os endereços de contacto estão listados no website do EIZO, abaixo.

<http://www.eizoglobal.com>

## Oplysninger om genbrug

Dette produkt forventes ved bortskaffelse at blive indsamlet og genbrugt i overensstemmelse med lovgivningen i dit land for at reducere belastningen af miljøet. Når du bortskaffer denne produkt, skal du kontakte en distributør eller et tilknyttet selskab i dit land.

Adresserne på kontaktpersonerne er angivet på EIZO's websted nedenfor.

<http://www.eizoglobal.com>

## Kierrätystä koskevia tietoja

Tuote tulee hävittää kierrättämällä maan lainsäädännön mukaisesti ympäristön kuormittumisen vähentämiseksi. Kun hävität tuotteen, ota yhteyttä jälleenmyyjään tai tytäryhtiöön maassasi.

Yhteystiedot löytyvät EIZOn Internet-sivustolta.

<http://www.eizoglobal.com>

## Wykorzystanie surowców wtórnych

Ten produkt po zużyciu powinien być zbierany i przetwarzany zgodnie z krajowymi przepisami dotyczącymi ochrony środowiska. Wyrzucając ten produkt, należy skontaktować się z lokalnym dystrybutorem lub partnerem. Adresy kontaktowe można znaleźć we wskazanej poniżej witrynie internetowej firmy EIZO.

<http://www.eizoglobal.com>

## Informace o recyklaci

Při likvidaci produktu musí být produkt vyzvednut a recyklován podle zákonů příslušné země, aby nedocházelo k zatěžování životního prostředí. Zbavujete-li se produktu, kontaktujte distributora nebo pobočku ve své zemi. Kontaktní adresy jsou uvedeny na následující webové stránce společnosti EIZO.

<http://www.eizoglobal.com>

## Ringlussevõtu alane teave

Keskkonnakoormuse vähendamiseks tuleks kasutatud tooted kokku koguda ja võtta ringlusse vastavalt teie riigi seadustele. Kui kõrvaldate käesoleva toote, võtke palun ühendust turustaja või filiaaliga oma riigis.

Kontaktisikud on toodud EIZO veebilehel järgmisel aadressil:

<http://www.eizoglobal.com>

## Újrahasznosítási információ

Jelen terméket, amikor eldobjuk, az illető ország törvényei alapján kell összegyűjteni és újrahasznosítani, hogy csökkentsük a környezet terhelését. Amikor ezt a terméket eldobja, kérjük lépjen kapcsolatba egy országon belüli forgalmazóval vagy társvállalkozással.

A kapcsolat címei az EIZO alábbi weboldalon találhatók felsorolva:

<http://www.eizoglobal.com>

## Podatki o recikliranju

Ob koncu uporabe odslužen izdelek izročite na zbirno mesto za recikliranje v skladu z okoljevarstveno zakonodajo vaše države. Prosimo, da se pri odstranjevanju izdelka obrnete na lokalnega distributerja ali podružnico.

Kontaktne naslovi so objavljeni na spodaj navedenih spletnih straneh EIZO:

<http://www.eizoglobal.com>

## Informácie o recyklácii

Pri likvidácii musí byť tento výrobok recyklovaný v súlade s legislatívou vašej krajiny, aby sa znížilo zaťaženie životného prostredia. Keď sa chcete zbaviť tohto výrobku, kontaktujte prosím distribútora alebo pobočku vo vašej krajine.

Zoznam kontaktných adries je uvedený na nasledovnej webovej stránke firmy EIZO:

<http://www.eizoglobal.com>

## Pārstrādes informācija

Utilizējot šo produktu, tā savākšana un pārstrāde veicama atbilstoši jūsu valsts likumdošanas prasībām attiecībā uz slodzes uz vidi samazināšanu. Lai veiktu šī produkta utilizāciju, sazinieties ar izplatītāju vai pārstāvniecību jūsu valstī.

Kontaktadresses ir norādītas zemāk minētajā EIZO tīmekļa vietnē.

<http://www.eizoglobal.com>

## Informacija apie grąžinamąjį perdirbimą

Šalinant (išmetant) šiuos gaminius juos reikia surinkti ir perdirbti grąžinamuoju būdu pagal jūsų šalies teisės aktus, siekiant sumažinti aplinkos teršimą. Kai reikia išmesti šį gaminį, kreipkitės į jūsų šalyje veikiančią gaminių pardavimo atstovybę arba jos filialą.

Kontaktiniai adresatai nurodyti EIZO interneto svetainėje.

<http://www.eizoglobal.com>

## Информация относительно рециклиране

При изхвърлянето на този продукт се предлага събирането и рециклирането му съобразно законите на вашата страна за да се намали замърсяването на околната среда. Когато искате да се освободите от този продукт, моля свържете се с търговския му представител или със съответните органи отговарящи за това във вашата страна.

Данните за връзка с нас са описани на следния Интернет сайт на EIZO:

<http://www.eizoglobal.com>

## Informație referitoare la reciclare

Acest produs, când debarasat, trebuie colectat și reciclat conform legislației țării respective ca să reducem sarcina mediului. Când se debarasează de acest produs, vă rugăm să contactați un distribuitor sau un afiliat al țării respective.

Adresele de contact le găsiți pe pagina de web al EIZO:

<http://www.eizoglobal.com>

## معلومات عن اعادة التدوير

عند التخلص من هذا المنتج، من الضروري تجميعه واعادة تدويره وفقاً للقوانين المتبعة في بلدك لتقليل العبء المسلط على البيئة. عندما تتخلص من هذا المنتج، يرجى الاتصال بموزع أو طرف منتسب إليه في بلدك. عناوين الاتصال مدرجة في موقع ايزو EIZO ادناه على شبكة الانترنت.

<http://www.eizoglobal.com>

## Geri Dönüşüm Bilgisi

Bu ürünün, atılacağı zaman, ülkenizin çevre kirliliğinin azaltılması konusundaki mevzuatına göre toplanması ve yeniden değerlendirilmesi gerekmektedir. Ürünü atacağınız zaman lütfen ülkenizdeki bir distribütör veya ilgili kuruluşla temasa geçiniz.

İrtibat adresleri aşağıdaki EIZO web sitesinde verilmiştir.

<http://www.eizoglobal.com>

