

Stand Mounting Instructions for the RadiForce® LS560W, LS580W, LS580W JP and LX600W

The FST5600 stand is designed for the RadiForce® 8MP large screen monitor series. Please ensure that all monitor weight and stability requirements are met before using the stand with other models (VESA mount 400mm x 400mm).


Order No.: 6GF69888BA01


Description: FST5600

A. Installing the FST5600 stand on LS560W and LS580W monitors

Step 1

Insert the 4 M8x20 screws approx.
5 mm deep into the designated holes at
the rear of the display.

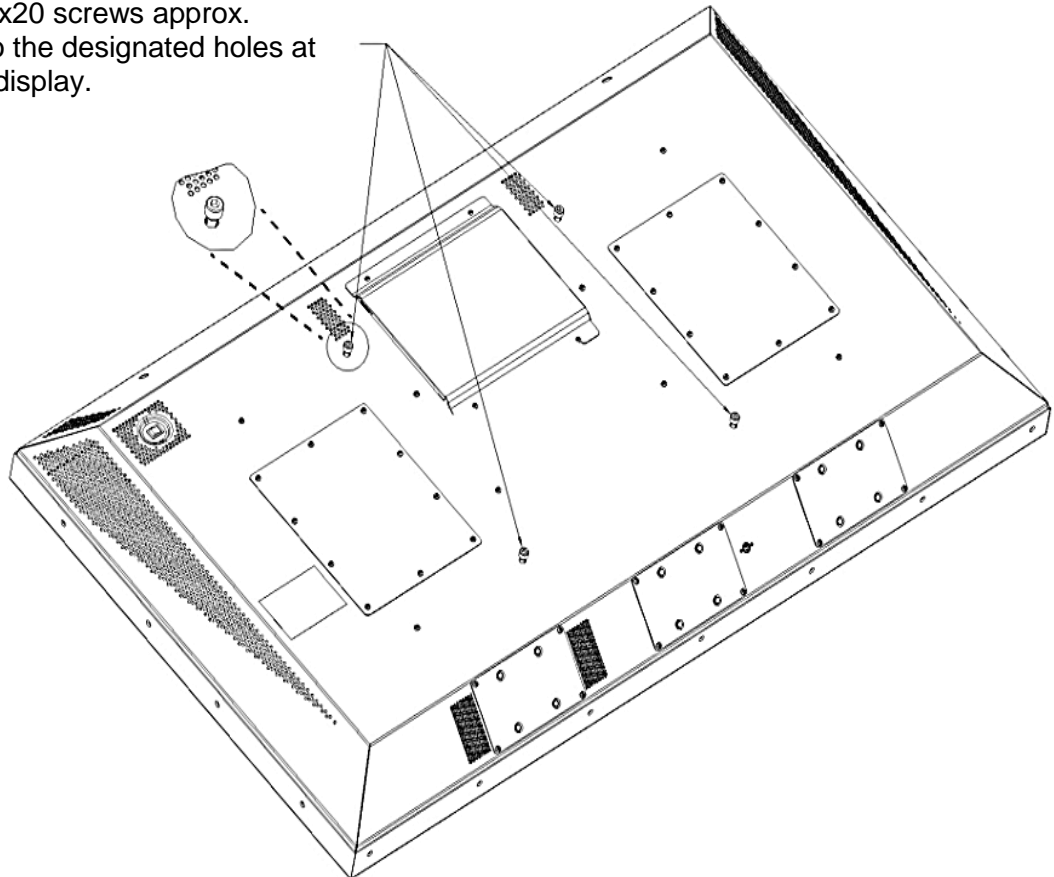

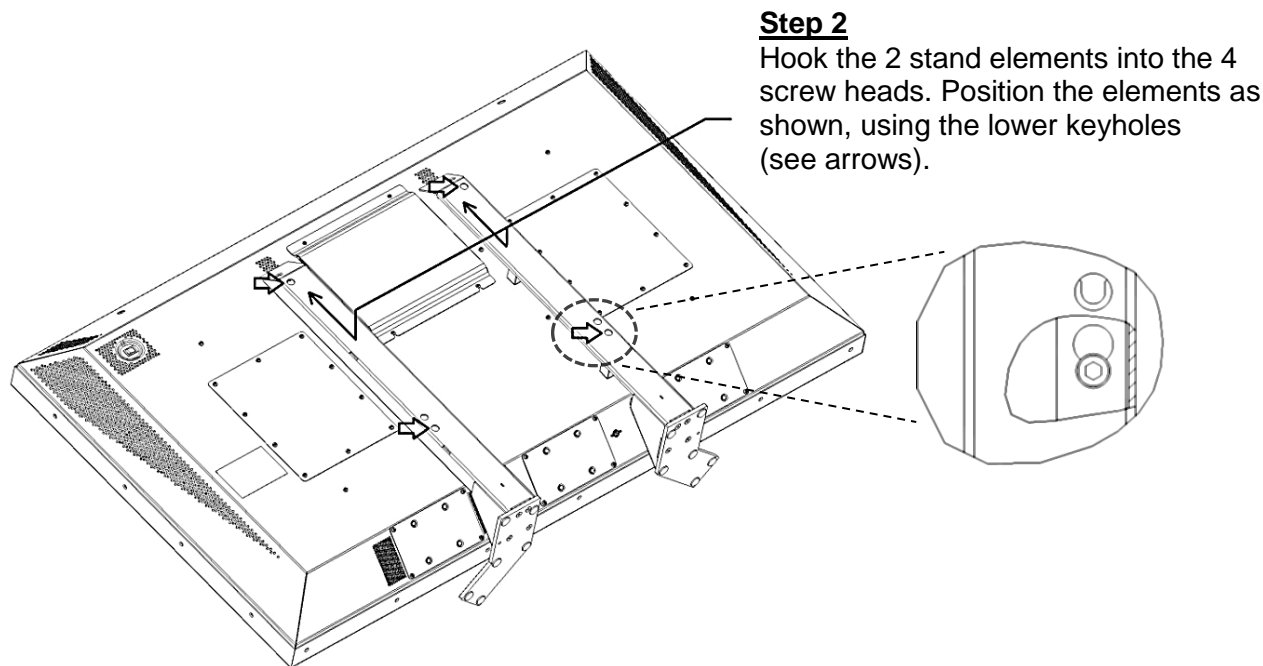


Step 3
Tighten the 4 screws with an Allen wrench (18 Nm).

Step 4
Place the monitor in its final location.



Caution


- The device must be removed from its packaging and carried by at least 2 persons.
- Wear appropriate protection gear to prevent injuries should the device be dropped.

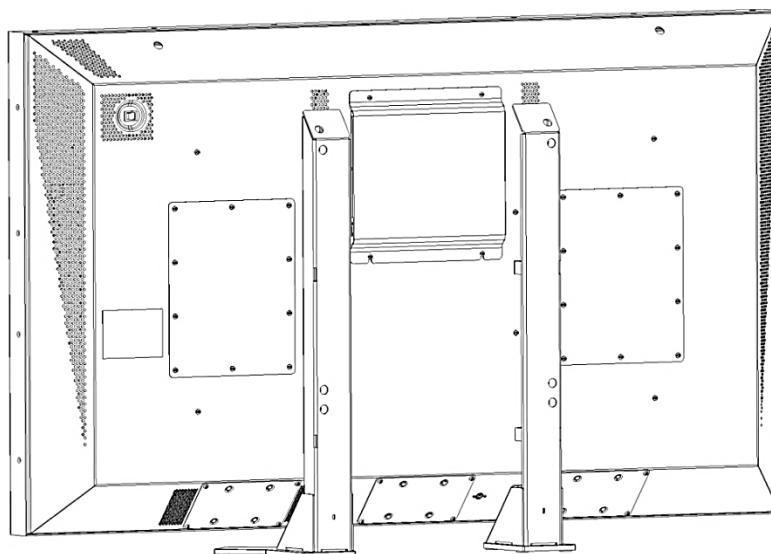


B. Installing the FST5600 stand on LX600W monitors

Step 1

Insert the 4 M8x20 screws approx.
5 mm deep into the designated holes at
the rear of the display.

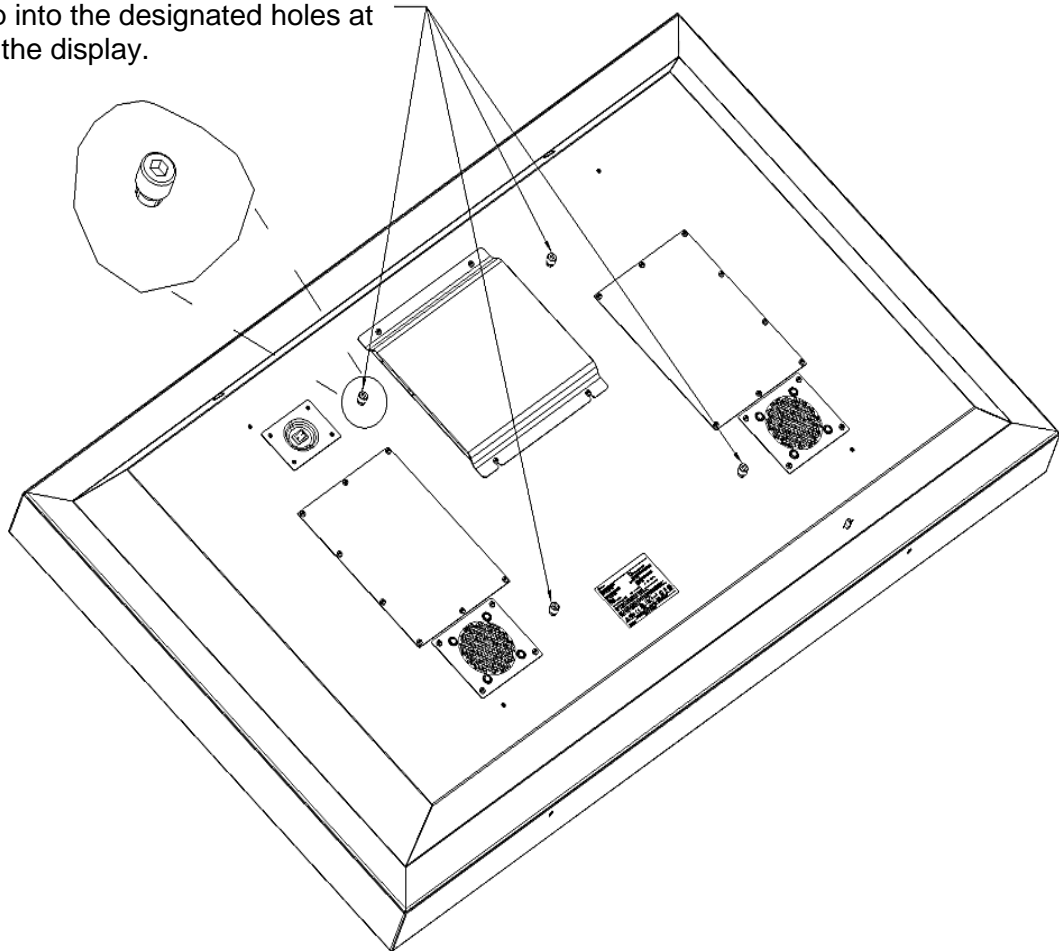

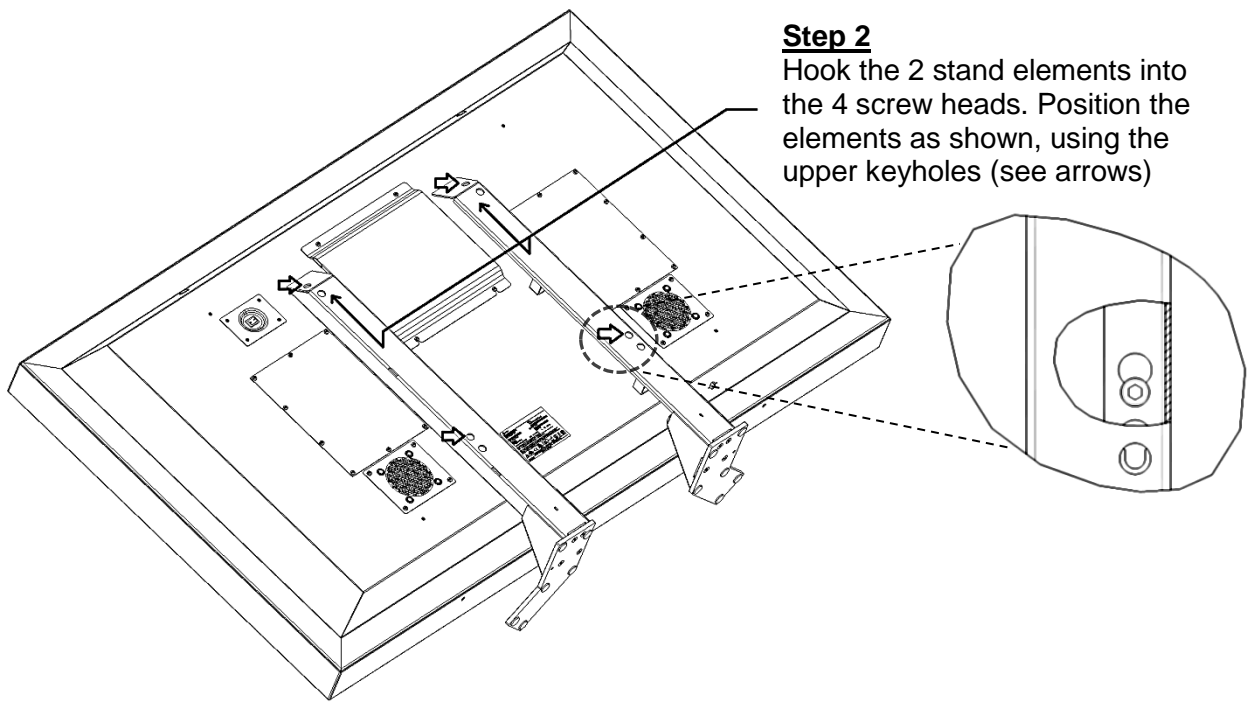

Step 3

Tighten the 4 screws with an Allen wrench (18 Nm)

Step 4

Place the monitor in its final location



Caution

- The device must be removed from its packaging and carried by at least 2 persons.
- Wear appropriate protection gear to prevent injuries should the device be dropped.

