Instructions for Use

CuratOR® EX3220-3D

Important

Please read this "Instructions for Use", and "Installation Manual" (separate volume) carefully to familiarize yourself with safe and effective usage.

Please retain this manual for future reference.

For monitor adjustment and settings, refer to the "Installation Manual".



SYMBOLS

This manual and this product use the symbols below. They denote critical information. Please read them carefully.

	WARNING		CAUTION
\wedge	Failure to abide by the information in a	\triangle	Failure to abide by the information in a
\	WARNING may result in serious injury and can be life-threatening.		CAUTION may result in moderate injury and/or property or product damage.
	and can be me-uneatering.		property or product damage.
\triangle	Indicates a warning or caution. For example, 🛕 indicates an "electrical shock" hazard.		
	Indicates a prohibited action. For example, 🌑 means "Do not disassemble".		
0	Indicates a mandatory action. For example, (means "Ground the unit".		

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.


No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to keep any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

PRECAUTIONS

IMPORTANT

- This product has been adjusted specifically for use in the region to which it was originally shipped. If the product is used outside the region, it may not operate as specified in the specifications.
- To ensure personal safety and proper maintenance, please read carefully this section and the caution statements on the monitor.

Location of the Caution Statements

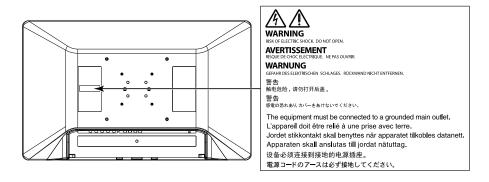

Symbols on the unit

Symbol		This symbol indicates	
Ċ	Power Switch:	Press to turn the monitor's power off.	
•	Power Switch:	Press to turn the monitor's power on.	
	Direct current		
A	Alerting to electrical hazar	rd	
\triangle	CAUTION		
\bigvee	Potential equalization term	ninal	
A	WEEE marking:	Product must be disposed of separately; materials may be recycled.	
C€	CE marking:	EU conformity mark in accordance with the provisions of Council Directive and/or Regulation (EU).	
	Manufacturer		
~~ <u> </u>	Date of manufacture		
RXonly	Caution: Federal law (USA licensed healthcare practi	A) restricts this device to sale by or on the order of a tioner.	
EU Medical Device	Medical device in EU		
EU Importer	Importer in EU		
UK CA	Mark signifying compliance with UK regulations		
UK Responsible Person	UK Responsible Person		
CH REP	Authorised representative in Switzerland		
EC REP	Authorised representative in the European Community		

WARNING

If the unit begins to emit smoke, smells like something is burning, or makes strange noises, disconnect all power connections immediately and contact your EIZO representative for advice.

Attempting to use a malfunctioning unit may result in fire, electric shock, or equipment damage.

Do not disassemble or modify the unit.

Opening the cabinet or modifying the unit may result in fire, electric shock, or burns.



Use multiple units or have ready a standby unit.

Prepare an appropriate countermeasure in case the monitor fails.

Do not turn the bushing to fix the AC adapter power cable.

Doing so may result in fire, electric shock, or equipment damage.



Refer all servicing to qualified service personnel.

Do not attempt to service this product yourself as opening or removing covers may result in fire, electric shock, or equipment damage.

Keep small objects or liquids away from the unit.

Small objects accidentally falling through the ventilation slots into the cabinet or spillage into the cabinet may result in fire, electric shock, or equipment damage. If an object or liquid falls/spills into the cabinet, unplug the unit immediately. Have the unit checked by a qualified service engineer before using it again.



Install the unit correctly on a sturdy and stable location using an arm or stand.

In accordance with the User Manual of each product, install it correctly on a sufficiently sturdy desk or wall. If the unit is installed incorrectly, it may drop or fall over, resulting in personal injury or equipment damage. If the unit falls, disconnect the power immediately and ask your local EIZO representative for advice. Do not continue using a damaged unit. Using a damaged unit may result in fire or electric shock.

Use the unit in an appropriate location.

Otherwise, fire, electric shock, or equipment damage may result.

- · Do not place outdoors.
- Do not place in any form of transportation (ships, aircraft, trains, automobiles, etc.).
- · Do not place in dusty or humid environments.
- Do not place in locations where water may be splashed on the screen (bathrooms, kitchens, etc.)
- Do not place in locations where smoke or steam come in direct contact with the screen.
- Do not place near heat generating devices or humidifiers.
- Do not place in locations where the product is subject to direct sunlight.
- Do not place in environments with flammable gas.
- Do not place in environments with corrosive gases (such as sulfur dioxide, hydrogen sulfide, nitrogen dioxide, chlorine, ammonia, ozone, etc.)
- Do not place in environments with dust, components that accelerate corrosion in the atmosphere (such as sodium chloride and sulfur), conductive metals, etc.



Use the enclosed power cord and connect to the standard power outlet in your country.

Be sure to use within the rated voltage of the power cord. Otherwise, fire or electric shock may result. Power supply: 100-240Vac 50/60Hz





Use the enclosed AC adapter.

The enclosed AC adapter (FSP120M-KAA) is for use with this product only. Do not use the AC adapter with other equipment. Do not use an AC adapter designed for other devices with this product.

Connecting to power sources that do not match the power ratings of the AC adapter may result in fire or electric shock.

To disconnect the power cord or adapter power cable, grasp the plug firmly and pull.

Tugging on the cord or cable may damage it and result in fire or electric shock.



OK



The equipment must be connected to a grounded main outlet.

Failure to do so may result in fire or electric shock.




Use the correct voltage.

- The unit is designed for use with a specific voltage only. Connection to another voltage than that specified in this
 "Instructions for Use" may cause fire, electric shock, or equipment damage.
 Power supply: 100-240Vac 50/60Hz
- Do not overload your power circuit, as this may result in fire or electric shock.

Handle the power cord and AC adapter with care.

Handle the power cord and AC adapter with care.

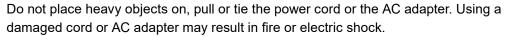



The operator should not touch the patient while touching the product.

This product has not been designed to be touched by patients.



Never touch the plug, AC adapter or power cord during a thunderstorm.

Touching them may result in electric shock.



Do not touch a damaged LCD panel directly with bare hands.

Liquid crystal is poisonous. If any part of your skin comes in direct contact with the panel, wash thoroughly. If liquid crystal enters your eyes or mouth, immediately flush with large amounts of water and seek medical attention.



A CAUTION

Check the operational state before use.

- · Begin use after ensuring that there are no problems with the displayed image.
- When rotating the display direction of the input image by 180° or flipping it horizontally, check the image and status icon before use.
- · When using multiple units, begin use after ensuring that the images are displayed appropriately.
- When viewing 3D images, use 3D polarized anti-fog glasses, and ensure that the environmental requirements for viewing 3D images are satisfied.

Securely fix cables / cords that have a fixing feature.

If they are not fixed securely, cables / cords may disconnect, and subsequently images may be cut off and your operations may be disrupted.

Handle with care when carrying the unit.

Disconnect the power cord and cables when moving the unit. Moving the unit with the power cord or cables attached is dangerous and may result in injury.


Carry or place the unit according to the correct specified methods.

• Monitors of size 30 inches and above are heavy. When unpacking and/or carrying the monitor, ensure at least two people are involved.

Dropping the unit may result in injury or equipment damage.

Do not block the ventilation slots on the cabinet.

- · Do not place any objects on the ventilation slots.
- Do not install the unit in a place with poor ventilation or inadequate space.
- Do not use the unit laid down or upside down.

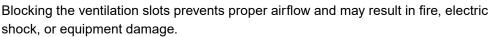



Do not touch the plug or AC adapter with wet hands.

Doing so may result in electrical shock.



Use an easily accessible power outlet.

This is to facilitate disconnecting the power in case of a problem.

The AC adapter becomes hot during use.

- Do not cover or place anything on top of the AC adapter. Do not place the AC adapter on top of things that trap heat such as carpets, blankets, etc. Keep the AC adapter away from direct sunlight and heat sources. Failure to do so may result in fire.
- Before moving the monitor, be sure to turn off the power switch, disconnect the power plug from the power outlet, and wait until it has cooled completely.

Do not suspend the AC adapter in midair.

Using the adapter while it is hanging suspended may result in fire or electrical shock.



Do not place the AC adapter in a vertical orientation.

Otherwise, dust or water may enter the adapter and may result in fire or electrical shock.



Do not subject the unit and the AC adapter to any impact due to dropping them or any other causes.

Using the adapter after it has been subjected to impact may result in fire or electrical shock.





Do not subject the LCD panel to strong impact.

Otherwise, glass will break and may result in injury.



Do not use the 3D polarized anti-fog glasses for any purpose other than for viewing 3D images.



Doing so may result in eye damage.

Periodically clean the area around the power plug and the ventilation slot of the monitor and the AC adapter.

Dust, water, or oil on the plug may result in fire.

Unplug the unit before cleaning it.

Cleaning the unit while it is plugged into a power outlet may result in electric shock.

If you plan to leave the unit unused for an extended period of time, disconnect the power plug from the wall socket after turning off the power switch for safety and power conservation.

For users in the territory of the EEA and Switzerland:

Any serious incident that has occurred in relation to the device should be reported to the Manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

About the monitor

Intended Use

This product is intended to be used to display (3D/2D) medical images, such as endoscopic surgery images.

Attention

- This product is not intended for diagnostic purposes.
- This product should be set to horizontal view mode.
- This product may not be covered by warranty for uses other than those described in this manual.
- The specifications stipulated in this manual are only applicable when the enclosed power cord is used.
- · Only use optional products manufactured or specified by us with this product.

Precautions for Use

- Parts (such as the LCD panel) may deteriorate with long-term use. Periodically check that they are
 operating normally.
- When the screen image is changed after displaying the same image for extended periods of time, an
 afterimage may appear. Use the screen saver or power save function to avoid displaying the same image
 for extended periods of time.
- If the monitor displays continuously over a long period of time, dark smudges or burn-in may appear. To maximize the life of the monitor, we recommend the monitor be turned off periodically.
- An afterimage may appear even after a short period has elapsed depending on the displayed image. If this occurs, changing the image or leaving the power off for a few hours may solve the problem.
- The backlight of the LCD panel has a fixed lifetime. When the screen becomes dark or begins to flicker, please contact your local EIZO representative.
- The screen may have defective pixels or a small number of light dots on the screen. This is due to the characteristics of the panel itself, and is not a malfunction of the product.
- Do not press on the panel or edge of the frame strongly, as this may result in display malfunctions, such as interference patterns, etc. If pressure is continually applied to the panel, it may deteriorate or damage it. (If the pressure marks remain on the panel, leave the monitor with a black or white screen. The symptom may disappear.)
- Do not scratch or press on the panel with any sharp objects, as this may result in damage to the panel.

 Do not attempt to wipe with tissues as this may scratch the panel.
- When the monitor is cold and brought into a room or the room temperature goes up quickly, dew
 condensation may occur on the interior and exterior surfaces of the monitor. In the same way, if the
 monitor is moved from a room with low temperature to a room with high temperature, dew condensation
 may form. In that case, do not turn the monitor on. Instead wait until the dew condensation disappears,
 otherwise it may cause some damage to the monitor.
- It takes about 60 minutes for the performance of the electronic parts within this product to stabilize.

 Please wait 60 minutes or more after the monitor power has been turned on or the monitor has recovered from power saving mode before making adjustments to it.

Cleaning

Periodic cleaning is recommended to keep the monitor looking new and to prolong its operation lifetime. Gently wipe off any dirt on the cabinet or panel surface with a soft cloth soaked in a small amount of water or one of the chemicals listed below.

Chemicals that may be used for cleaning

Material name	Product name
Ethanol	Ethanol
Isopropyl alcohol	Isopropyl alcohol
Benzalkonium chloride	Welpas
Glutaral	Sterihyde
Glutaral	Cidex Plus28
Ammonia	Ammonia water
Hydrogen peroxide	Hydrogen peroxide solution
Alkyldiaminoethylglycine hydrochloride	Satenidin solution
Benzalkonium chloride	Zalkonin solution
Benzethonium chloride	Bezeton solution

Attention

- Do not use chemicals on a frequent basis. Chemicals such as alcohol and antiseptic solution may cause gloss variation, tarnishing, and fading of the cabinet or panel, and also quality deterioration of the image.
- · Never use any thinner, benzene, wax, or abrasive cleaner, which may damage the cabinet or panel.
- Do not let chemicals come into direct contact with the monitor.

About Cleaning the 3D polarized anti-fog glasses

Since the anti-fog treatment on the inner side of the lens makes it susceptible to damage, wipe the lens gently with your finger using a soft cloth or gauze soaked in a small amount of ethanol.

How to use the monitor comfortably

- Look at the screen from a proper distance and from a proper angle.
- If you become fatigued or experience discomfort while viewing 3D images, stop using the monitor.

CONTENTS

PRECA	AUTIONS 3		
IMPORTANT 3			
About the monitor8			
Inte	nded Use8		
Prec	autions for Use8		
Clea	ning9		
How	to use the monitor comfortably9		
CONTE	ENTS10		
Chapte	er 1 Introduction11		
1-1.	Features11		
1-2.	Package Contents12		
1-3.	Controls and Functions12		
Chapte	er 2 Installation / Connection16		
2-1.	Before Installing the Product16		
	Installation Requirements16		
2-2.	Installing the Product17		
2-3.	Connecting the Power Cord18		
2-4.	Connecting the Cables19		
2-5.	Installing the Cable Cover 20		
2-6.	Turning On the Power20		
Chapte	er 3 If No Image Is Displayed21		
Chapte	er 4 Specifications22		
4-1.	Specifications List22		
4-2.	Displayable Input Signals (2D) 24		
4-3.	Displayable Input Signals (3D) 25		
4-4.	Optional Accessories25		
Appendix26			
Med	Medical Standard26		
EMC Information27			
Warning for Badia interference 22			

Chapter 1 Introduction

1-1. Features

High-quality and high-resolution Full HD LCD panel

- The IPS panel enables high-resolution, high-quality medical images to be displayed.
- · Equipped with front protection panel
- · LED Backlight
- · Optical bonding for high visibility

Supports 3D display

• The Xpol® polarized filter is installed on the LCD panel, enabling a stereoscopic view with 3D polarized anti-fog glasses.

(Xpol® is a registered trademark of Arisawa Mfg. Co., Ltd.)

- · Supports major 3D image input formats.
 - SIDE BY SIDE
 - LINE BY LINE
 - TOP AND BOTTOM
 - SIMUL
- Parallax adjustment function (3D display only)
 Binocular parallax adjustment is possible during 3D display.

Supports multiple I/O

- 3G-SDI signal: 2 inputs / 2 outputs (Also supports dual-stream system)
- DVI signal: 2 inputs / 1 output

Simultaneous input of two types of HD images is possible.

- · Component / Analog RGB signal (BNC): 1 input / 1 output
- S VIDEO signal: 1 input / 1 output
- · Composite video signal: 1 input / 1 output
- Analog RGB signal (D-Sub): 1 input

Other functions

· 2 screen display function

Permits parallel display of two input images.

Two modes, P in P and P out P, are available.

Allows switching of display size.

· Display direction switching function

The display direction of the input image can be switched.

The image can be rotated 180° or flipped horizontally.

Display the status icon according to the display direction (180°: $\boxed{3}$, flipped horizontally: $\boxed{9}$).

· Gamma switch function

Gamma function is installed according to usage.

Simplified DICOM® image is supported.

· Color temperature switch function

Color temperature switch function is installed according to usage.

- External remote function
 - Remote control using RS-232C or GPI terminal.
- Direct input signal switching can be assigned to a function button.
- IP32 protection structure (Excluding the AC adapter)

 The IPx2 protection level is effective when the monitor is installed so it cannot be put at a slant.

1-2. Package Contents

Check that all of the following items are included in the package.


Note

- It is recommended that the box and packing materials be stored so that they can be used to move or transport this product.
 - Monitor
 - · Power cord
 - AC adapter (FSP120M-KAA)
 - · Cable cover (with screws)


- · Monitor attachment screw
 - (M4×12) x 4
 - (M6×15) x 4
- 3D polarized anti-fog glasses x 3
- User Manual CD
- · Instructions for Use

1-3. Controls and Functions

AC adapter

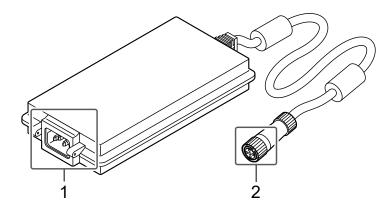


1. AC IN terminal	Connects the power cord.
2. DC OUT terminal Connect to the DC IN terminal on the monitor.	

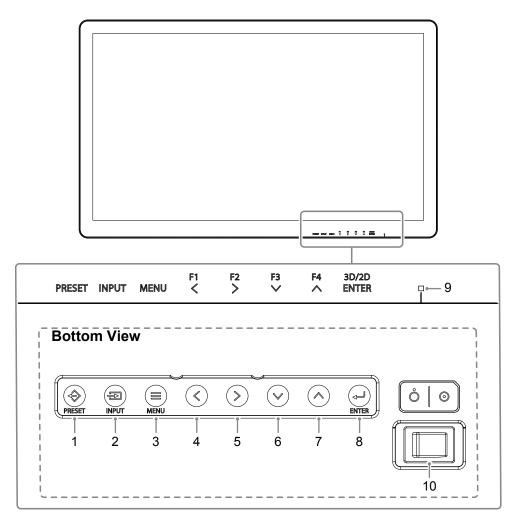


1. (PRESET) button	Displays the preset menu.
2. (INPUT) button	Displays the input select menu.
3. (MENU) button	Displays the main menu.
4. ((F1) button	Executes the function assigned to this button. Select items in the menu screen.
5. (F2) button	
6. 🕑 (F3) button	
7. 🛆 (F4) button	
8. (3D/2D, ENTER) button	Switches the 3D and 2D displays. Select items in the menu screen.
9. Power indicator	The indicator color differs depending on the monitor's operation status.
	Green: Monitor in operation, Orange: Power saving mode, Off: Power off
10. Power switch	Turns the power on or off.
	⊙ : On, ံ : Off

Rear

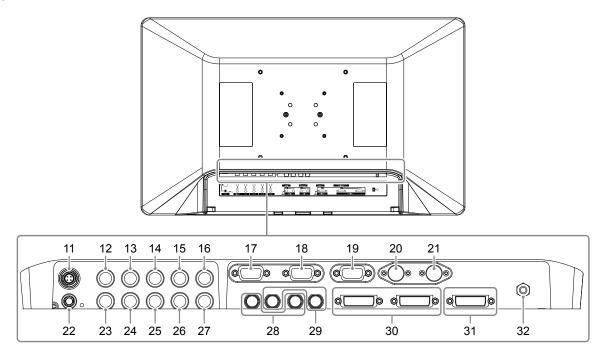

11. DC IN terminal	Connects the DC OUT terminal of the AC adapter.
12. VIDEO input terminal	Connects devices with Composite video output terminals.
(BNC type)	' '
13. Y/G input terminal	Connects devices with Y (luminance) for component output, or devices with G for
(BNC type)	RGB component.
14. P _B /B input terminal	Connects devices with P _B /C _B (blue differential) for component output, or devices
(BNC type)	with B for RGB output.
15. P _R /R input terminal	Connects devices with P _R /C _R (red differential) for component output, or devices with
(BNC type)	R for RGB output.
16. SYNC input terminal	When this product is operated in external synchronization, it connects devices with
(BNC type)	C.SYNC standard signal output.
17. GPI input terminal	Connects to GPI-supported external devices. By assigning functions to each
(D-Sub 9pin)	terminal, this product can be controlled by external devices.
18. RS-232C terminal	Controls this product by connecting it to an external device.
(D-Sub 9pin)	Input switching and various adjustments are possible from connected external
	devices.
19. HD15 input terminal	Connects devices with a synchronous signal with Analog RGB output such as
(D-Sub15 pin (mini))	computers.
20. S VIDEO input terminal	Connects devices with S VIDEO output terminals.
(4pin mini DIN)	
21. S VIDEO output terminal	The signal input to S VIDEO input terminal is output as is.
(4pin mini DIN)	

Rear (Continued)

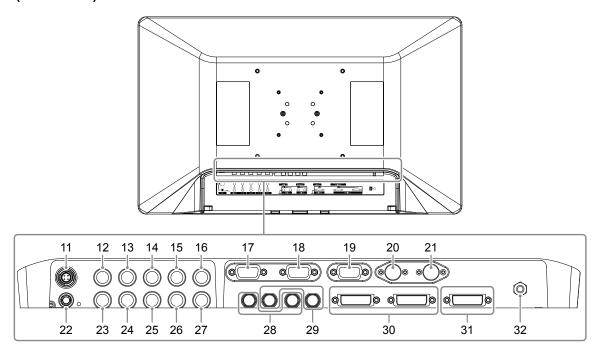

22.	DC OUT terminal	Used to supply 5V power to a peripheral.
		Attention
		No device coming in contact with a patient may be connected to the DC OUT
		terminal.
23.	VIDEO output terminal	The signal input to VIDEO input terminal is output as is.
	(BNC type)	
24.	Y/G output terminal	The signal input to Y/G input terminal is output as is.
	(BNC type)	
25.	P _B /B output terminal	The signal input to P _B /B input terminal is output as is.
	(BNC type)	
26.	P _R /R output terminal	The signal input to P _R /R input terminal is output as is.
	(BNC type)	
27.	SYNC output terminal	The signal input to SYNC input terminal is output as is.
	(BNC type)	
28.	SDI 1/2 input terminal	Connects devices with SDI output terminals.
	(BNC type)	
29.	SDI 1/2 output terminal	The signal input to SDI 1/2 input terminal is output as is.
	(BNC type)	
30.	DVI-D 1/2 input terminal	Connects devices with DVI-D output.
	(DVI-D)	
31.	DVI-D 2 output terminal	The signal input to DVI-D 2 input terminal is output as is.
	(DVI-D)	
32.		This terminal ensures equal potential between the monitor and other devices.
	terminal	Connects equipotential plugs.

Note

• VIDEO, Y/G, P_B/B , and P_R/R input terminals, when not connected to any output terminal, are terminated with internal 75 Ω . If a cable is connected to the output terminal, the internal terminal automatically opens. When a cable connected to the output terminal is not connected to a video device, and when the connected video device is not terminated with 75 Ω , the signal level becomes too large and cannot be displayed correctly.

Chapter 2 Installation / Connection

2-1. Before Installing the Product

Carefully read "PRECAUTIONS" (page 3) and always follow the instructions.

When installing this product, perform thorough operational testing (of the system, cables, arms, etc.) in the environment where the product will be used.

Installation Requirements

When installing the monitor, ensure that there is adequate space around the sides, back, top, and bottom of the monitor.

Attention

- Position the monitor so that there is no light to interfere with the screen.
- Do not use any materials or objects that will cover the monitor or the AC adapter.

2-2. Installing the Product

This product should be installed using an arm or stand.

Attention


- · When installing, do so by carefully following the information in the User Manual about the arm or stand.
- Ensure the following and select components that comply with the VESA standards.
 - Clearance between the screw holes: 100 mm \times 100 mm, 200 mm \times 200 mm
 - Strong enough to support weight of the monitor unit (excluding the stand) and attachments such as cables.
- Use the supplied screws (M4 screws for 100 mm x 100 mm, M6 screws for 200 mm x 200 mm) when installing the product.
- When using an arm or stand, attach it to achieve the following tilt angles of the monitor.
 - Up 45°, down 45°
- · Connect the cables after attaching an arm or stand.
- The monitor and other attached parts are heavy. Dropping them may result in injury or damage to equipment.
- Periodically check the tightness of the screws. If not sufficiently tight, the monitor may detach from the arm, which may result in injury or equipment damage.

1. Attach the arm or stand to the back of the monitor by aligning the four screw holes and secure the arm or stand using the screws supplied with the monitor.


Screw tightening torque: 1.0 N·m to 1.4 N·m (M4 screws), 1.5 N·m to 2.0 N·m (M6 screws)

Tool required: Phillips screwdriver (#2)


Rear

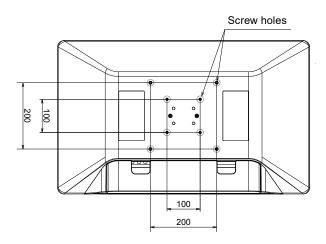

Using the supplied screws

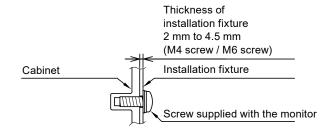

Using commercially available screws

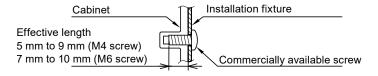


2-3. Connecting the Power Cord


Attention

- Turn off the monitor before connecting it.
- When removing the power cord, always remove the power plug from the power outlet first.
- 1. Connect the power cord to the AC IN terminal on the AC adapter.


Insert the power cord all the way to the back.

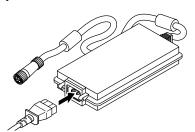

2. With the notch (shown in the figure) of the DC OUT terminal connector of the AC adapter facing you, insert it straight into the DC IN terminal of the monitor.

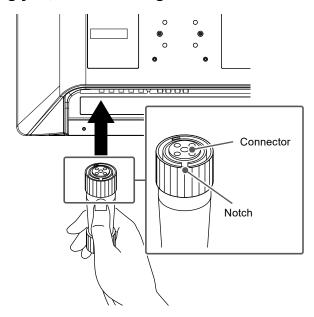


3. Turn the lock ring clockwise to secure it in place.

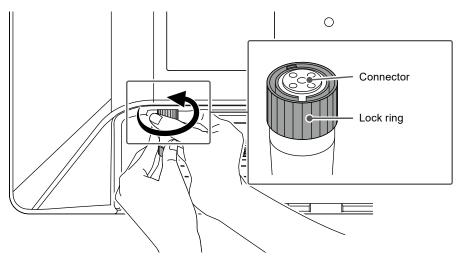

4. Check the rated value on the AC adapter and connect the power plug to the power outlet.

Attention

• When positioning the AC adapter vertically, do not place it with the AC inlet facing up.

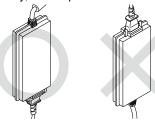

OK: Horizontal position

NOK: Vertical position

· Secure the adapter using a banding band such as a cable tie as necessary to prevent it from dropping.

2-4. Connecting the Cables

1. Connect the cables appropriate for the device to be used.

Attention

- · Do not use damaged cables.
- Do not connect or disconnect the signal cable while the monitor is turned on.
- The terminal parts of the SDI terminal and DVI-D terminal are vulnerable to static electricity, so be sure to work with care during installation. When working with the monitor, be sure to observe the following:
 - Do not touch the connector pins.
 - Do not touch pins at the end of any cable connected to a connector.
 - Take anti-static precautions such as using an anti-static wrist strap when working.


Note

• The warning label is displayed as shown in the right-hand diagram by the SDI terminal and DVI-D terminal.



2-5. Installing the Cable Cover

- 1. Align the cable cover on the back of the monitor in such a way that the cables can go through the cable outlet port.
- 2. Insert the cable cover tabs into the grooves in the monitor.
- 3. Tighten the screws at the left and right-hand holes on the bottom side of the monitor.

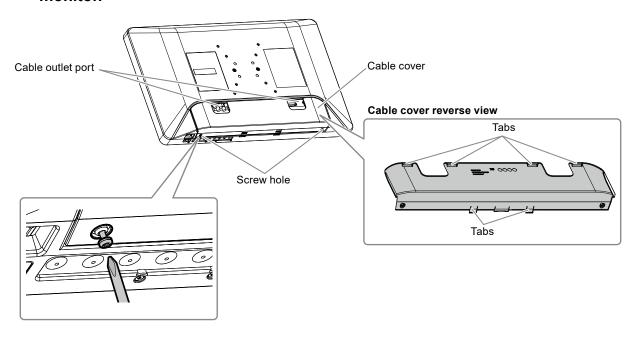

Attention

- Ensure that cables are not pinched between the cable cover and the monitor.
- Securely tighten the screws in the two locations. (Screw tightening torque: 0.4 N·m to 0.7 N·m, Tool required: Phillips screwdriver (#2))
- Avoid subjecting the terminal and cable to stress.
- Do not pack or transport with the cable cover attached.

2-6. Turning On the Power

1. Turn on the power switch on the bottom of the monitor, and then turn on the monitor.

The power indicator on the front of the monitor lights up green.

If the indicator does not light up, see "Chapter 3 If No Image Is Displayed" (page 21).

Note

• If the power switch on the bottom of the monitor is turned off, the monitor is turned off.

Chapter 3 If No Image Is Displayed

Problem	Possible cause and remedy
1. No picture	 Check whether the power cord is connected properly. Check whether the DC OUT terminal and DC IN terminal are connected properly. Turn on the power switch. Turn off the power and then turn it on again.
2. The message below appears.	This message appears when the signal is not input correctly even though the monitor is functioning properly.
This message appears when no signal is input. Example: SDI 1 No Signal	 The message shown on the left may appear, because some devices to be connected do not output the signal immediately after power-on. Check whether the device to be connected is turned on. Check whether the signal cable is connected properly. Turn off the power and then turn it on again.
The message indicates that the input signal is outside the specified frequency range. Example: DVI-D 1 Not Supported	 Check whether the device to be connected is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "4-2. Displayable Input Signals (2D)" (page 24), "4-3. Displayable Input Signals (3D)" (page 25)). Reboot the device to be connected.

Chapter 4 Specifications

4-1. Specifications List

Monitor

Type	LCD Panel			
Size	Туре	Color (IPS)		
Resolution (H x V) 1920 x 1080 1920 y 1080 1920 x 1080 x	Backlight	LED		
Display Size (H x V) 698 mm × 393 mm Pixel Pitch 0.364 mm Display Colors 8-bit: 16.77 million colors 10-bit (SDI only): 1073.74 million colors (Max.) Viewing Angles (H / V, typical) Pirghtness (typical) 700 cd/m² Response Time (typical) 1200:1 Video Signals Input Terminals DVI (DVI-D) × 2 Single link, HDCP support SDI (BNC) × 2 3G / HD / SD-SDI Analog RGB (B-C) (D-Sub 15 pin (mini)) × 1 Component / Analog RGB (BNC × 4) × 1 Ps/B: 0.7 Vp-p, 75 Ω (including synchronous signals) Pg/B: 0.7 Vp-p, 75 Ω (Ps/B-C) Ps/B-C) SVIDEO (4 pin mini DIN) × 1 Single link, DVI-D (HDCP unsupported) SDI (BNC) × 2 3G / HD / SD-SDI Analog RGB (BNC × 4) × 1 Ps/B: 0.7 Vp-p, 75 Ω (including synchronous signals) Pg/B: 0.7 Vp-p, 75 Ω (Ps/B-C) Ps/B-C) Ps/B-C	Size	80.1 cm (31.5 inch)		
Pixel Pitch Display Colors S-bit: 16.77 million colors 10-bit (SDI only): 1073.74 million colors (Max.)	Resolution (H x V)	·		
Sebit: 16.77 million colors	Display Size (H x V)	698 mm × 393 mm		
10-bit (SDI only): 1073.74 million colors (Max.) Viewing Angles (H / V, typical) 178" / 178" 178"	Pixel Pitch	0.364 mm		
Viewing Angles (H / V, typical) 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 178 ' 179 ' 179 ' 170 cd/m²	Display Colors	8-bit: 16.77 million colors		
H / V, typical Brightness (typical) 700 cd/m² 8 ms (Gray-to-gray) 200:1		10-bit (SDI only): 1073.74 million	on colors (Max.)	
Brightness (typical) 700 cd/m² 8 ms (Gray-to-gray)		178° / 178°		
Response Time (typical) 1200:1	, ,			
Video Signals				
DVI (DVI-D) × 2 Single link, HDCP support		, , , , , , , , , , , , , , , , , , , ,		
Input Terminals	() ,	1200:1		
SDI (BNC) × 2 3G / HD / SD-SDI				
Analog RGB	Input Terminals			
(D-Sub 15 pin (mini)) × 1		` '		
$ \begin{array}{c ccccccccccccccccccccccccccccccccccc$				
$ \begin{array}{c ccccccccccccccccccccccccccccccccccc$				
$\begin{array}{c ccccccccccccccccccccccccccccccccccc$				
Sync: 0.3 Vp-p to 4.0Vp-p, 75 Ω		(BNC × 4) × 1		
			1	
$S \ VIDEO (4 \ pin \ mini \ DIN) \times 1 $ Brightness signal: $1.0 \ Vp-p, 75 \ \Omega$ Color signal: $0.286 \ Vp-p, 75 \ \Omega$ Color signal: $0.286 \ Vp-p, 75 \ \Omega$ Single link, DVI-D (HDCP unsupported)		Composite video (BNC) × 1		
$\begin{tabular}{ c c c c c } \hline Color signal: 0.286 \ Vp-p, 75 \ \Omega \\ \hline Output Terminal & DVI (DVI-D) \times 1 & Single link, DVI-D (HDCP unsupported) \\ \hline SDI (BNC) \times 2 & 3G \ / \ HD \ / \ SD-SDI \\ \hline Component \ / \ Analog RGB & Y/G: 1.0 \ Vp-p, 75 \ \Omega (including synchronous signals) \\ \hline (BNC \times 4) \times 1 & P_B/B: 0.7 \ Vp-p, 75 \ \Omega \\ \hline P_R/R: 0.7 \ Vp-p, 75 \ \Omega \\ \hline Sync: 0.3 \ Vp-p \ to 4.0 \ Vp-p, 75 \ \Omega \\ \hline Sync: 0.3 \ Vp-p \ to 4.0 \ Vp-p, 75 \ \Omega \\ \hline Sync: 0.3 \ Vp-p \ to 4.0 \ Vp-p, 75 \ \Omega \\ \hline Color signal: 0.286 \ Vp-p, 75 \ \Omega \\ \hline Color signal: 0.286 \ Vp-p, 75 \ \Omega \\ \hline Color signal: 0.286 \ Vp-p, 75 \ \Omega \\ \hline \hline \textbf{Monitor Control Terminal} & RS-232C \ (D-Sub 9 \ pin) \times 1 \\ \hline \textbf{GPI (D-Sub 9 \ pin)} \times 1 \\ \hline \textbf{Power} \\ \hline Input & DC \ 24 \ V \pm 10 \ \%, 3.96 \ A \\ \hline Maximum Power \\ \hline Consumption & Max. 95 \ W \\ \hline \end{tabular}$, ,	* * * *	
$\begin{tabular}{cccccccccccccccccccccccccccccccccccc$			1	
$\begin{tabular}{ c c c c c c c c c c c c c c c c c c c$	Output Terminal	DVI (DVI-D) × 1	Single link, DVI-D (HDCP unsupported)	
$\begin{array}{c ccccccccccccccccccccccccccccccccccc$		SDI (BNC) × 2	3G / HD / SD-SDI	
$\begin{array}{c ccccccccccccccccccccccccccccccccccc$		Component / Analog RGB	Y/G: 1.0 Vp-p, 75 Ω (including synchronous signals)	
$Sync: 0.3 \text{ Vp-p to } 4.0 \text{ Vp-p, } 75 \Omega$ $Composite \text{ video (BNC)} \times 1 \qquad 1.0 \text{ Vp-p, } 75 \Omega$ $S \text{ VIDEO (4 pin mini DIN)} \times 1 \qquad Brightness signal: 1.0 \text{ Vp-p, } 75 \Omega$ $Color \text{ signal: } 0.286 \text{ Vp-p, } 75 \Omega$		(BNC × 4) × 1		
$S \text{ VIDEO (4 pin mini DIN)} \times 1 \text{Brightness signal: } 1.0 \text{ Vp-p, } 75 \Omega \\ \text{Color signal: } 0.286 \text{ Vp-p, } 75 \Omega \\ \hline \text{Monitor Control} \\ \hline \text{Monitor Control Terminal} \text{RS-232C (D-Sub 9 pin)} \times 1 \\ \hline \text{GPI (D-Sub 9 pin)} \times 1 \\ \hline \text{Power} \\ \hline \text{Input} \text{DC } 24 \text{ V} \pm 10 \text{ \%, } 3.96 \text{ A} \\ \hline \text{Maximum Power} \\ \hline \text{Consumption} \text{Max. } 95 \text{ W}$				
Monitor Control Monitor Control Terminal RS-232C (D-Sub 9 pin) x 1 GPI (D-Sub 9 pin) x 1 Power Input DC 24 V ± 10 %, 3.96 A Maximum Power Consumption Max. 95 W		· , ,		
Monitor Control Monitor Control Terminal RS-232C (D-Sub 9 pin) x 1 GPI (D-Sub 9 pin) x 1 Power Input DC 24 V ± 10 %, 3.96 A Maximum Power Consumption Max. 95 W		S VIDEO (4 pin mini DIN) x 1		
Monitor Control Terminal RS-232C (D-Sub 9 pin) x 1 GPI (D-Sub 9 pin) x 1 Power Input DC 24 V ± 10 %, 3.96 A Maximum Power Consumption Max. 95 W	Marita a Carata a L		Color signal: 0.286 Vp-p, 75 Ω	
GPI (D-Sub 9 pin) x 1 Power Input DC 24 V ± 10 %, 3.96 A Maximum Power Consumption Max. 95 W		TD0 0000 (D 0 1 0 :) 4		
Power Input DC 24 V ± 10 %, 3.96 A Maximum Power Consumption Max. 95 W	Monitor Control Terminal	` '		
Input DC 24 V ± 10 %, 3.96 A Maximum Power Max. 95 W Consumption		GPI (D-Sub 9 pin) x 1		
Maximum Power Max. 95 W Consumption		1		
Consumption	•			
DC OUT terminal 5 V, 1 A		Max. 95 W		
	DC OUT terminal	5 V, 1 A		

Physical Specifications			
External dimensions (W × H × D)	760 mm × 463 mm × 96 mm		
Net Weight	Approx. 10.8 kg		
Protection structure	IP32 (The IPx2 protection level is effective when the monitor is installed so it cannot be put at a slant.)		
Operating Environmental Requirements			
Temperature	0 °C to 35 °C (32 °F to 95 °F)		
Humidity	20 % to 85 % R.H. (no condensation)		
Air Pressure	540 hPa - 1060 hPa		
Transportation / Storage En	Transportation / Storage Environmental Requirements		
Temperature	-20 °C to 60 °C (-4 °F to 140 °F)		
Humidity	10 % to 90 % R.H. (no condensation)		
Air Pressure	540 hPa to 1060 hPa		

AC adapter

Power		
Input	100 - 240 VAC, 47 - 63 Hz, 1.6 - 0.8 A	
Maximum Power	Max. 106 W	
Consumption		
Physical Specifications		
External dimensions	175.2 mm × 90.2 mm × 28.2 mm	
$(W \times H \times D)$		
Net Weight	Approx. 760 g	
Operating Environmental Re	equirements	
Temperature	0 °C to 35 °C (32 °F to 95 °F)	
Humidity	20 % to 85 % R.H. (no condensation)	
Air Pressure	540 hPa - 1060 hPa	
Transportation / Storage Environmental Requirements		
Temperature	-20 °C to 60 °C (-4 °F to 140 °F)	
Humidity	10 % to 90 % R.H. (no condensation)	
Air Pressure	540 hPa to 1060 hPa	

4-2. Displayable Input Signals (2D)

√: Supported

	Horizontal	Vertical			RGB /	RGB / Y		
Signal name	Frequency	Frequency	VIDEO	SDI 1	YP_BP_R	HD1		DVI 1
orginal manne	(kHz)	(Hz)	S VIDEO	SDI 2	YP_BP_R	RGB-VIDEO	RGB-PC	DVI 2
	` '	` ′			mode	mode	mode	
NTSC	15.734	59.940	$\sqrt{}$	-	-	-	-	-
PAL	15.625	50.000	$\sqrt{}$	-	-	-	-	-
480 / 60i	15.734	59.940	-	$\sqrt{}$		$\sqrt{}$	$\sqrt{}$	$\sqrt{}$
480 / 60p	31.469	59.940	-	-	√	√	√*2	$\sqrt{}$
576 / 50i	15.625	50.000	-	√	$\sqrt{}$	√	√	√
576 / 50p	31.250	50.000	-	-		√	√	$\sqrt{}$
720 / 60p	45.000	60.000	-	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$
720 / 50p	37.500	50.000	-	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$	$\sqrt{}$
1080 / 60i	33.750	60.000	-	√	√	√	V	√
1080 / 60p	67.500	60.000	-	√	√	√	√*2	V
1080 / 50i	28.125	50.000	-	√	√	√	1	√
1080 / 50p	56.250	50.000	-	√	√	√	V	V
1080 / 30p	33.750	30.000	-	√	√	√	V	√
1080 / 25p	28.125	25.000	-	√	√	√	V	√
1080 / 24p	27.000	24.000	-	√	√	√	V	√
1080 / 24PsF	27.000	48.000	-	√	√	√	V	-
720 x 400@70Hz	31.469	70.087	-	-	-	√	V	-
640 x 480@60Hz	31.469	59.940	-	-	-	√*1		
640 x 480@72Hz	37.861	72.809	-	-	-	√	√	-
640 x 480@75Hz	37.500	75.000	-	-	-	√		-
640 x 480@85Hz	43.269	85.008	-	-	-	√	V	-
800 x 600@56Hz	35.156	56.250	-	-	-	√	V	-
800 x 600@60Hz	37.879	60.317	-	-	-	√	V	√
800 x 600@72Hz	48.077	72.188	-	-	-	√	V	-
800 x 600@75Hz	46.875	75.000	-	-	-	√	V	-
800 x 600@85Hz	53.674	85.061	-	-	-	√	V	-
1024 x 768@60Hz	48.363	60.004	-	-	-	√*3	√*3	√
1024 x 768@70Hz	56.476	70.069	-	-	-	√	V	-
1024 x 768@75Hz	60.023	75.029	-	-	-	√	V	-
1024 x 768@85Hz	68.677	84.997	-	-	-	√	1	-
1152 x 864@75Hz	67.500	75.000	-	-	-	√	V	-
1280 x 768@60Hz	47.776	59.870	-	-	-	√*3	√*3	√
1280 x 800@60Hz	49.702	59.810	-	-	_	√	V	√
1280 x 960@60Hz	60.000	60.000	-	-	-	√	V	√
1280 x 1024@60Hz	63.981	60.020	-	-	-	√	V	√
1280 x 1024@75Hz	79.976	75.025	-	-	-	√	1	-
1400 x 1050@60Hz	65.317	59.978	-	-	-	√*3	√*3	√
1440 x 900@60Hz	55.935	59.887	-	-	-	√	V	V
1680 x 1050@60Hz	65.290	59.954	-	-	-	√*3	√*3	√
1600 x 1200@60Hz	75.000	60.000	-	-	-	√*3	√*3	√
1920 x 1080@60Hz	67.500	60.000	-	-	-	√*1	√	√*4
1920 x 1200@60Hz	74.038	59.950	-	-	-	√*3	√*3	V

^{*1} If a 640 × 480@60Hz signal is input and RGB-VIDEO mode is selected, it is recognized as 480/60p. If a 1920 × 1080@60Hz signal is input and RGB-VIDEO mode is selected, it is recognized as 1080/60p.

"4:3": 1024 × 768@60Hz, 1600 × 1200@60Hz, 1400 × 1050@60Hz

"Wide": 1280 × 768@60Hz, 1920 × 1200@60Hz, 1680 × 1050@60Hz

^{*2} If a 480/60p signal is input and RGB-PC mode is selected, it is recognized as 640 × 480@60Hz. If a 1080/60p signal is input and RGB-PC mode is selected, it is recognized as 1920 × 1080@60Hz.

^{*3} When the resolution of the input signal is different from that of the screen display, change the setting value of the signal format (analog adjustment).

^{*4} A 1920 x 1080@60Hz signal of DVI-D is recognized as 1080/60p.

Attention

• Input Sync on Green or composite Sync (VBS) as sync signal into the RGB input terminal. Some PCs cannot display images.

Note

• The input signal above is a displayable input signal when using a single product without connecting an output.

4-3. Displayable Input Signals (3D)

√: Supported

Signal name	Horizontal Frequency (kHz)	Vertical Frequency (Hz)	SDI 1 SDI 2	DVI-D 1 DVI-D 2
720 / 60p	45.000	60.000	$\sqrt{}$	$\sqrt{}$
720 / 50p	37.500	50.000	$\sqrt{}$	
1080 / 60i	33.750	60.000	$\sqrt{}$	√
1080 / 60p	67.500	60.000	$\sqrt{}$	$\sqrt{}$
1080 / 50i	28.125	50.000	$\sqrt{}$	$\sqrt{}$
1080 / 50p	56.250	50.000	$\sqrt{}$	$\sqrt{}$
1080 / 30p	33.750	30.000	$\sqrt{}$	$\sqrt{}$
1080 / 25p	28.125	25.000	$\sqrt{}$	$\sqrt{}$
1080 / 24p	27.000	24.000	$\sqrt{}$	
1080 / 24PsF	27.000	48.000		-
1024 x 768@60Hz	48.363	60.004	-	√ ·
1280 x 1024@60Hz	63.981	60.020	-	√

4-4. Optional Accessories

The following accessories are available separately.

3D polarized anti-fog glasses	H3G01
Stand	HST02

Appendix

Medical Standard

- It is necessary to ensure that the final system is in compliance with IEC60601-1 requirements.
- Power-supplied equipment can emit electromagnetic waves, that could influence, limit or result in malfunction of the monitor. Install the equipment in a controlled environment, in which such effects are avoided.

Classification of Equipment

- Electric shock protection type: Class I
- EMC class: IEC60601-1-2 Group 1 Class B
- Medical device classification (EU): Class I
- Mode of operation: Continuous
- IP class: IP32 (The IPx2 protection level is effective when the monitor is installed so it cannot be put at a slant.)

EMC Information

The EX3220-3D has the capability to displays medical images properly.

Environments of Intended Use

The EX3220-3D is intended to be used in the following environments.

• Professional healthcare facility environments such as clinics and hospitals (including use in the vicinity of high-frequency surgical equipment such as electrosurgical knives).

The following environments are not suitable for using the EX3220-3D.

- · Home healthcare environments
- In the vicinity of short-wave therapy equipment
- · RF shielded room of the medical equipment systems for MRI
- In shielded location Special environments
- · Installed in vehicles including ambulances
- · Other special environments



WARNING

The EX3220-3D requires special precautions regarding EMC and need to be installed. You need to carefully read EMC Information and the "PRECAUTIONS" section in this document, and observe the following instructions when installing and operating the product.

The EX3220-3D should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

When using a portable RF communication equipment, keep it 30 cm (12 inches) or more away from any part, including cables, of the EX3220-3D. Otherwise, degradation of the performance of this equipment could result.

Anyone who connects additional equipment to the signal input part or signal output parts, configuring a medical system, is responsible that the system complies with the requirements of IEC60601-1-2.

Images may be distorted if the product is used near a device such as high-frequency surgical equipment. Check in advance to ensure that no problems occur during use.

Do not touch the signal input/output connectors while using the EX3220-3D. It may affect the displayed image.

Be sure to use cables that satisfy the following requirements.

Use of cables that do not satisfy the requirements could result in increased electromagnetic emissions, reduced electromagnetic immunity of this equipment, and incorrect operation.

Cables	Max. Cable Length	Shielding
AC Cord	2 m	Unshielded
DC Cord	17.5 m	Shielded
BNC Cable (SDI)	30 m	Shielded
DVI Cable	5 m	Shielded
BNC Cable (Composite)	5 m	Shielded
D-Sub Cable	5 m	Shielded
S VIDEO Cable	5 m	Shielded
RS-232C Cable	5 m	Shielded

Technical Specifications

Electromagnetic emissions

The EX3220-3D is intended for use in the electromagnetic environment specified below.

The customer or the user of the EX3220-3D should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment - Guidance	
RF emissions CISPR11	Group 1	The EX3220-3D uses RF energy only for its internal function. Therefore, its RF emission are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR11	Class B	The EX3220-3D is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network	
Harmonic emissions IEC61000-3-2	Class D	that supplies buildings used for domestic purposes.	
Voltage fluctuations / flicker emissions IEC61000-3-3	Complies		

Electromagnetic immunity

The EX3220-3D has been tested at the following compliance levels (C) according to the testing requirements (T) for Professional healthcare facility environments specified in IEC60601-1-2.

The customer or the user of the EX3220-3D should assure that EX3220-3D is used in the following environment.

Immunity test	Test level (T)	Compliance level (C)	Electromagnetic environment - Guidance
Electrostatic discharge (ESD) IEC61000-4-2	±8 kV contact discharge ±15 kV air discharge	±8 kV contact discharge ±15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transients / bursts IEC61000-4-4	±2 kV power lines ±1 kV input / output lines	±2 kV power lines ±1 kV input / output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surges IEC61000-4-5	±1 kV line to line ±2 kV line to ground	±1 kV line to line ±2 kV line to ground	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	$\begin{array}{c} 0 \ \% \ U_T (100 \ \% \ dip \ in \ U_T) \\ 0.5 \ cycles \ and \ 1 \ cycle \\ 70 \ \% \ U_T (30 \ \% \ dip \ in \ U_T) \\ 25 \ cycles \ at \ 50 \ Hz \\ 0 \ \% \ U_T (100 \ \% \ dip \ in \ U_T) \\ 250 \ cycles \ at \ 50 \ Hz \end{array}$	$0 \% \ U_T (100 \% \ dip \ in \ U_T)$ $0.5 \ cycles \ and \ 1 \ cycle$ $70 \% \ U_T (30 \% \ dip \ in \ U_T)$ $25 \ cycles \ at \ 50 \ Hz$ $0 \% \ U_T (100 \% \ dip \ in \ U_T)$ $250 \ cycles \ at \ 50 \ Hz$	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EX3220-3D requires continued operation during power mains interruptions, it is recommended that the EX3220-3D be powered from an uninterruptible power supply or a battery.
Power frequency magnetic fields IEC61000-4-8	30 A/m (50 / 60 Hz)	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. The product should be kept at least 15 cm away from the source of power frequency magnetic fields during use.

Electromagnetic immunity

The EX3220-3D has been tested at the following compliance levels (C) according to the testing requirements (T) for Professional healthcare facility environments specified in IEC60601-1-2.

The customer or the user of the EX3220-3D should assure that EX3220-3D is used in the following environment.

THE	The customer of the user of the EA3220-3D should assure that EA3220-3D is used in the following environment.				
lm	ımun			Electromagnetic environment -	
				(C)	Guidance
					Portable and mobile RF communications equipment should be used no closer to any part of the EX3220-3D, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance
		d distur-	3 Vrms 150 kHz - 80 MHz	3 Vrms	d = 1.2√P
RF f	ields 31000	duced by -4-6	130 KHZ - 60 WHZ		
			6 Vrms ISM bands between 150 kHz and 80 MHz ^{a)}	6 Vrms	d = 1.2√P
Rad	iated F	RF fields	3 V/m	3 V/m	d = 1.2√P, 80 MHz - 800 MHz
	61000		80 MHz - 2.7 GHz		d = 2.3√P, 800 MHz - 2.7 GHz
					Where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in meters (m).
					Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^{b)} , should be less than the compliance level in each frequency range ^{c)} .
	Interference may occur in the vicinity of equi marked with the following symbol.			Interference may occur in the vicinity of equipment marked with the following symbol.	
					((<u>*</u>))
Note	1	U_T is the a.	c. mains voltage prior to app	olication of the test level.	
Note	2	At 80 MHz	and 800 MHz, the higher fre	equency range applies.	
Note	3				RF fields or radiated RF fields may not apply in all and reflection from structures, objects and people.
a)					
b)	b) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EX3220-3D is used exceeds the applicable RF compliance level above, the EX3220-3D should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the EX3220-3D.				
(c)	c) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.				

Recommended separation distances between portable or mobile RF communication equipment and the EX3220-3D

The EX3220-3D is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the EX3220-3D can suppress electromagnetic interference by maintaining a minimum distance (30 cm) between portable and mobile RF communications equipment (transmitters) and the EX3220-3D. The EX3220-3D has been tested at the following compliance levels (C) for the required test levels (T) of immunity to the proximity electromagnetic fields in the following RF communication services.

Test frequency (MHz)	Bandwidth ^{a)} (MHz)	Service ^{a)}	Modulation b)	Test level (T) c) (V/m)	Compliance level (C) (V/m)
385	380 - 390	TETRA 400	Pulse modulation b) 18 Hz	27	27
450	430 - 470	GMRS 460, FRS 460	FM ±5 kHz deviation 1 kHz sine	28	28
710	704 - 787	LTE Band 13, 17	Pulse modulation b)	9	9
745	7		217 Hz		
780	7				
810	800 - 960	GSM 800 / 900,	Pulse modulation b)	28	28
870	7	TETRA 800,	18 Hz		
930		iDEN 820 CDMA 850, LTE Band 5			
1720	1700 - 1990	GSM 1800;	Pulse modulation b)	28	28
1845	7	CDMA 1900;	217 Hz		
1970		GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS			
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	28	28
5240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation b)	9	9
5500	7		217 Hz		
5785	7				
a) For some s	services, only the up	olink frequencies are inc	cluded.		•
b) The carrier	is modulated using	a 50 % duty cycle squa	are wave signal.		
 		h maximum nower and		stance	

c) Test levels were calculated with maximum power and 30 cm of separation distance.

The customer or the user of the EX3220-3D can help prevent interference caused by the proximity magnetic field by maintaining a minimum distance (15 cm) between RF transmitters and the EX3220-3D. The EX3220-3D has been tested at the following compliance level (C) for the requirement test level (T) of proximity magnetic field immunity.

Test frequency	Modulation	Test level (T) (A/m)	Compliance level (C) (A/m)
134.2 kHz	Pulse modulation ^{a)} 2.1 kHz	65	65
13.56 MHz	Pulse modulation ^{a)} 50 kHz	7.5	7.5
a) The carrier is modulat	ed using a 50 % duty cycle squ	are wave signal.	

The EX3220-3D is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. For other portable and mobile RF communication equipments (transmitters), minimum distance between portable and mobile RF communications equipment (transmitters) and EX3220-3D as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of	Separation distance according to frequency of transmitter (m)			
transmitter (W)	150 kHz – 80 MHz d = 1.2√P	80 MHz – 800 MHz d = 1.2√P	800 MHz – 2.7 GHz d = 2.3√P	
0.01	0.12	0.12	0.23	
0.1	0.38	0.38	0.73	
1	1.2	1.2	2.3	
10	3.8	3.8	7.3	
100	12	12	23	

For transmitters rated at a maximum output power not listed above, the recommended separation distance "d" in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1	At 80 MHz and 800 MHz, the separation distance for a higher frequency range must be applied.
Note 2	These guidelines regarding conducted disturbances induced by RF fields or radiated RF fields may not apply in all
	situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Warning for Radio interference

For U.S.A., Canada Only

FCC Supplier's Declaration of Conformity

We, the Responsible Party

5710 Warland Drive, Cypress, CA 90630

Phone: (562) 431-5011

Trade name: EIZO declare that the product

Model: CuratOR EX3220-3D

is in conformity with Part 15 of the FCC Rules. Operation of this product is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

WARNING!

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will required to correct the interference at his own expense.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note

Use the specified cable below so as to keep interference within the limits of a Class A digital device.

- AC Cord
- Shielded Signal Cable

Canadian Notice

This Class A information technology equipment complies with Canadian ICES-003. Cet équipement informatique de classe A est conforme à la norme NMB-003 du Canada.

For Australia, New Zealand, etc Only

Warning

Operation of this equipment in a residential environment could cause radio interference.

Warnung

Der Betrieb dieses Geräts in einer Wohnumgebung konnte Funkstörungen verursachen.

Avertissement

L'utilisation de cet équipement dans une zone résidentielle pourrait provoguer des interférences radio.



EIZO Corporation 153 Shimokashiwano, Hakusan, Ishikawa 924-8566 Japan

EIZO GmbH EC REP

Carl-Benz-Straße 3, 76761 Rülzheim, Germany

EIZO Limited UK Responsible Person

1 Queens Square, Ascot Business Park, Lyndhurst Road, Ascot, Berkshire, SL5 9FE, UK

EIZO AG

CH REP

Moosacherstrasse 6, Au, CH-8820 Wädenswil, Switzerland

KK C€

00N0N044J1 IFU-EX32203D-6