## Operating Instructions 3D Polarized Eyewear

Model No. **H3G01**

(For 3D Monitors CuratOR EX Series)

Before operating this product, please read the Operating Instructions carefully.  
Store this Operating Instructions in a safe place where they are accessible for operating personnel.

## Intended purpose

---

- This product “3D Polarized Eyewear” is used to view 3D images. This product may also be worn over standard prescription eye-glasses.

## Cautions

---

- Do not drop, exert pressure, or step on this product.  
(Doing so may result in injury and may damage this product.)
- Do not use this product if a malfunction or damage occurred.  
(Doing so may result in injury or eyesight fatigue.)
- Use the dedicated monitor by confirming Operating instructions of your monitor.  
(Use of improper combination may result in symptoms like nausea, dizziness or eyesight fatigue.)
- Stop using this product immediately if you feel skin irritations.  
(In rare cases, the materials used in this product may cause an allergic reaction.)
- Be careful of the hinge section and the tips of the frame when putting on this product.  
(Doing so may result in injury.)
- When using this product to view 3D images, please read the Operating Instructions of the monitor to be used.
- Do not use this product for purposes other than viewing 3D images.
- Keep your eyes and this product on a horizontal level to the monitor.  
(If you tilt the face, 3D effect is reduced, or the perception of colors may be distorted.)
- This product has an anti-fog coating on the inner side of the lenses. Fogging may occur due to perspiration and rapid temperature changes on non-coated lenses.

# Cleaning and storage

---

- Use a soft dry cloth intended for use on eye-glasses when cleaning this product. Remove lint from the cloth before use as it may damage the product. Do not use benzene, thinners, acid detergents, alkali detergents or similar solvents for cleaning as they could discolor the surface and cause the paint to peel.
- If the lenses of this product are soiled, wipe them with disinfectant ethanol moistened gauze and dry them.
- Do not apply water or other liquids to this product when cleaning.
- Avoid storing this product in places with a high temperature or high humidity.
- Please do not sterilize this product.

# Specifications

---


Dimensions (W × H × D)	: 155 mm × 48 mm × 165 mm (6.1" × 1.9" × 6.5")
Weight (Mass)	: 30 g (0.07 lb)
Ambient temperature	: 5 °C – 35 °C (41 °F – 95 °F)
Storage temperature	: -20 °C – 60 °C (-4 °F – 140 °F)
Storage humidity (Relative humidity)	: 5 % – 85 % (no condensation)
Type	: Circular Polarized Light
Material	: Nylon (frame), Triacetylcellulose (lens)

- Weight and dimensions when shown are approximately.

# Meaning of symbols

---

	Authorised representative in the European community
	Manufacturer

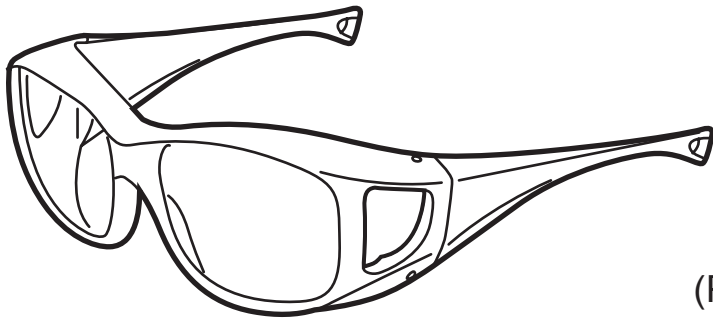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