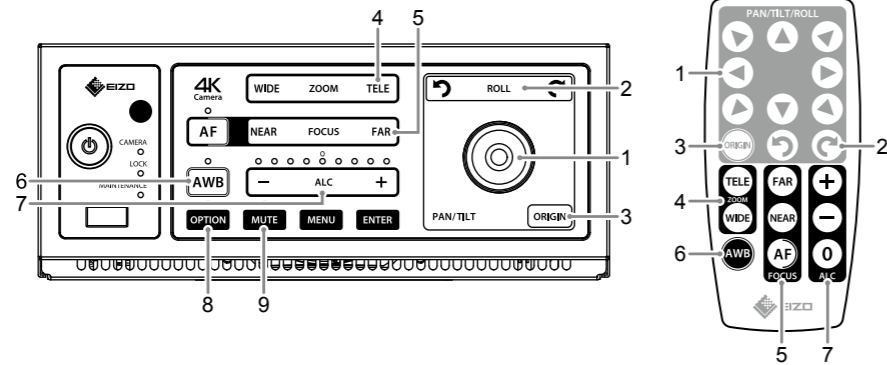
# CuratOR® SC430-PTR

## Quick Reference Guide **English**

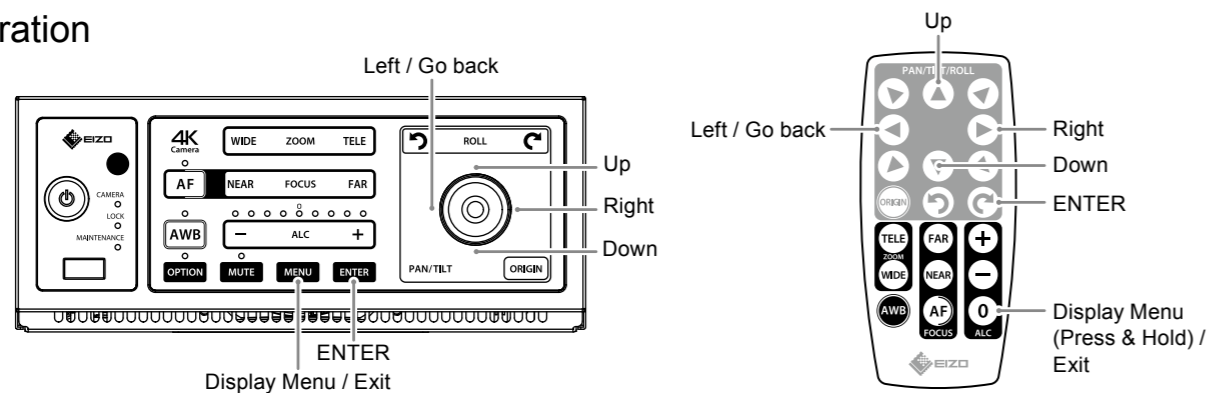
### Camera Control

**Attention** Do not move the drive unit of the camera with your hand. Be sure to use the CCU panel or the remote controller.

<b>1. Joystick</b>	Used for PAN/TILT movements of the camera. The camera moves to a direction to which the joystick is pushed. While the button on the remote controller is pressed down, the camera moves to the corresponding button direction.
<b>2. ROLL</b>	While ↺ is pressed, the camera rotates in a counterclockwise direction. While ↻ is pressed, the camera rotates in a clockwise direction.
<b>3. ORIGIN</b>	Hold down this button to restore the camera to the origin.
<b>4. ZOOM</b>	Used for zooming the camera. Keep pressing the "TELE" button to zoom in the telescopic direction. Keep pressing the "WIDE" button to zoom in the wide-angle direction.
<b>5. FOCUS</b>	Used for focusing the camera. Keep pressing the "FAR" button to focus farther. Keep pressing the "NEAR" button to focus closer. <ul style="list-style-type: none"> <li>Press the "AF" button to execute the OnePush AF.</li> <li>Hold down the "AF" button (approx. 2 sec) to switch to the real-time AF.</li> <li>Press the "FAR" or "NEAR" button to enter the manual focus mode. (The OnePush AF and real-time AF will be canceled.)</li> </ul> <p><b>Note</b></p> <ul style="list-style-type: none"> <li>When capturing an object located at a fixed distance, using the OnePush AF function is recommended in order to prevent the focus from changing.</li> </ul>
<b>6. AWB</b>	Used for automatically adjusting the color tint that is influenced by the light source color. Hold down the "AWB" button (approx. 2 sec) to keep the white balance at that point. When pressing this button, capture an achromatic object.
<b>7. ALC</b>	Used for adjusting the brightness level. Press the "+" / "-" button to change the image brightness. <ul style="list-style-type: none"> <li>Remote Controller <ul style="list-style-type: none"> <li>Press the "0" button to restore brightness level in the ALC mode to the initial state.</li> <li>Hold down the "0" button (approx. 2 sec) to display the menu.</li> </ul> </li> </ul>
<b>8. OPTION</b>	Used for accelerating the camera mount motion.
<b>9. MUTE</b>	Mutes audio output from the CCU.

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